OPT 40

MULTIFUNCTIONS PLUS

## CHI SIAMO

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## WHO WE ARE

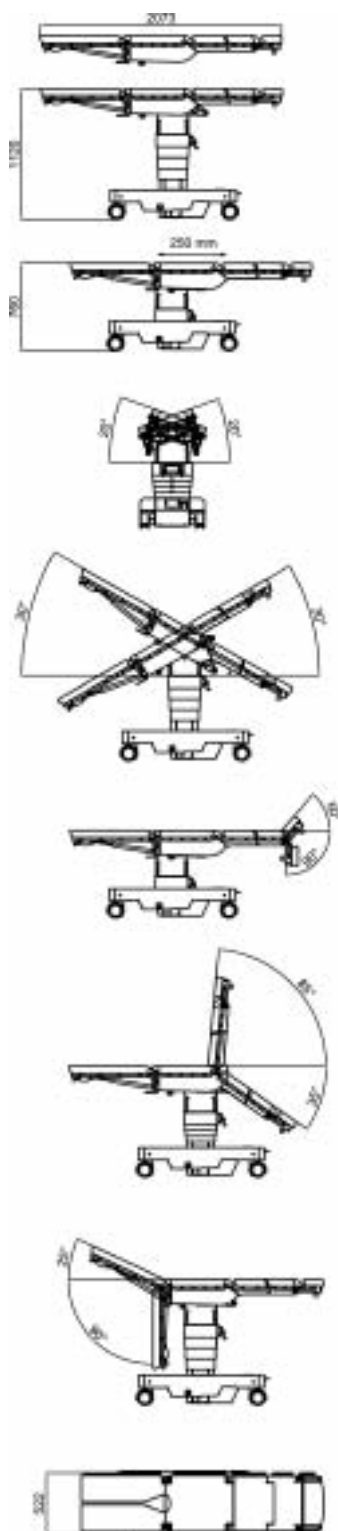
*OPT, Officina di Protesi Trento Spa, founded in 1921, has specialized in the manufacturing of operating tables, accessories and patient transfer units.*

*Working closely with surgeons, OPT has developed innovative products of high quality, safety and functionality. The significant resources invested in research and development are supported by state of the art CNC production lines, and rigorous testing controls on every unit.*

*The success and position OPT enjoys in both the ever-growing domestic and export markets underlines the high reliability characterizing*

*OPT's products. The OPT Quality System is accredited in accordance with*

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*The new operating table OPT 40/1 is up-to-date in its class as far as aesthetical and innovative criteria are concerned, it is functional in any detail, safe and versatile in any use. It has been designed and built in order to satisfy any requirement of the operating room and can be easily used in any surgery discipline.*

*A silent electro-hydraulic system that can be operated through an integrated control panel, cable linked control unit, I.R. control unit and foot pedal switch, allows 30° trendelenburg/antitrendelenburg, 20° bilateral inclination, 360 mm height adjustment going from a min height of 760 mm to a max height of 1120, 250 mm slide of tabletop particularly useful for radiology applications, back section tilting, flex/reflex and self-levelling of the operating table.*

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## Modular and Versatile

*A unique table top suitable for all surgical procedures, divided in 6 radiotransparent sections (standard configuration), interchangeable, complete with 60 mm soft, removable, radiotransparent and antistatic pads and with accessories' rails, section 25x10 mm, for the application of the specialist accessories:*

- Head-section tilting and removable.
- Lower back section electrically tiltable
- Removable upper back section, manually tiltable
- Pelvis section.
- Two separated leg-sections, each can be tilted, spread and removable, in two or four pieces.

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## Manoevrable but extremely stable

*The new OPT 40/1 has 4 large wheels guaranteeing an excellent manoeuvrability in every situation. An optional fifth wheel can be easily and rapidly raised/lowered by means of an innovative up/down foot command.*

*The stability of the table during the surgical procedures is guaranteed by four mechanical stabilisation pistons.*

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## User interface

On the base of the table there is a display for the self monitoring of the battery charge condition as well as the data transmission correct operating. The column is equipped with an integrated control panel to allow all the electrical movements.

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## Control units

All the electrical functions are independent from each other and can be operated through:

- Integrated control panel (standard in delivery).
- I.R. infrared control unit (optional).
- Cable linked control unit (optional).
- Foot pedal switch at 2 or 3 commands (optional).

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## Integration

OPT 40/1, through an optional system, guarantees the full compatibility with the integrated operating rooms developed by the major worldwide integrators.

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## Manual emergency override

An external device allows to perform manually the movements of the table in case of emergency.

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## Carbon fibre table top

OPT 40/1 is also available with the carbon fibre table top, particularly indicated for vascular, cardiac and ortho/trauma surgery, where 360° radiolucency is strictly needed.

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**SPINE SURGERY IN GENUPECTURAL POSITION**

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**SHOULDER SURGERY AND ARTHROSCOPY  
IN SEMISITTED DECUBITUS**

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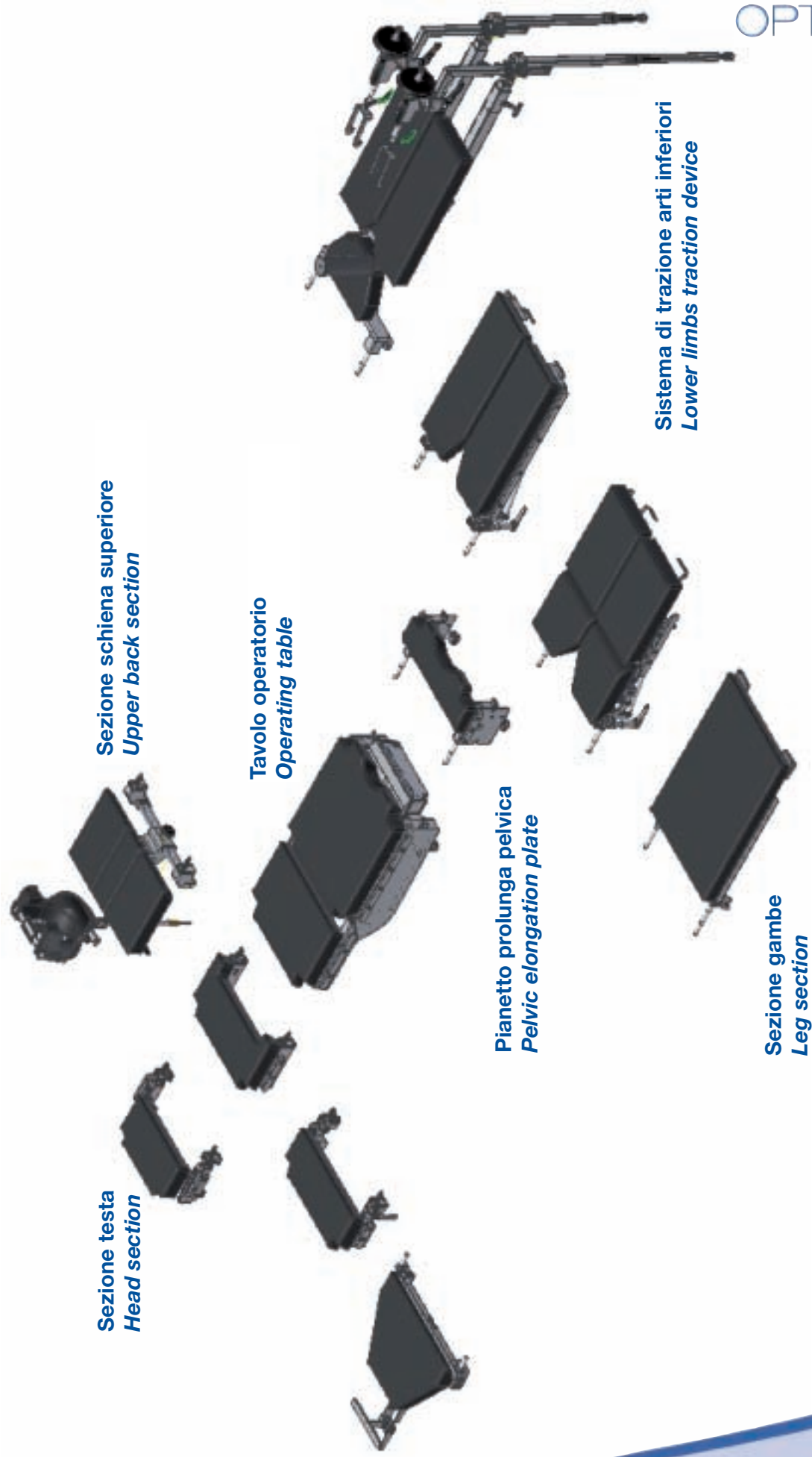
COMPANY  
WITH QUALITY SYSTEM  
CERTIFIED BY DNV  
=ISO 9001/2000=  
-ISO 13485-

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**OFFICINA DI PROTESI TRENTO SpA**
L'Officina di Protesi Trento SpA si riserva il diritto di apportare modifiche senza preavviso.
*Officina di Protesi Trento SpA reserves the right to make any modification without prior notice.*