

Dräger

Flow reduction

- Anti-air shower
- Manual disconnection

Manual disconnection Evita Infinity V500

Evita Infinity V500

When the **Manual disconnection** function is activated, Evita V500 **reduces flow delivery** in case of disconnection.

The patient can be disconnected within the next 10 seconds.

When the patient is reconnected, the device resumes ventilation in the set ventilation mode.

1. Touch the **Special procedures...** button in the main menu bar.
The device opens the Special procedures dialog window
2. Touch the **Functions** button if the page is not already preset
3. Touch the **Manual discon.** button (B) and
4. Confirm by pressing the **rotary knob**

The **Manual disconnection** maneuver can be configured as a **Manual discon.** button in the main menu bar to enable direct access.



Anti-air shower Evita Infinity V500

Flow reduction Anti Air Shower

If a disconnection is detected, flow delivery is reduced until after reconnection.

1. To activate manual disconnection touch **System setup ...**
2. Touch the **Ventilation** page button (A)
3. Touch the **General settings** tab (B)
4. Switch **Anti Air Shower** (H) on or off
5. Confirm by pressing the **rotary knob**

Evita V500 starts with the selected settings.



Dräger. Technology for Life®