

## Dräger Perseus® A500 Breathing System Disassembly/Assembly



### Disassembly

1 Unscrew the CLIC adapter.



Screw the CLIC adapter.

2 Insert the cleaning disk into the adapter. Close the adapter so that it engages.



Remove the cleaning disk from the adapter. Close the adapter so that it engages.

3 Loosen and pull out the expiration port.



Push in the expiratory port and tighten it with the knurled nut.

4 Loosen and pull out the inspiratory port.



Push in the inspiratory port and tighten it with the knurled nut.

5 Remove the flow sensors.



Insert the flow sensors.

6 Turn the three levers counterclockwise about 120°.



Turn the three levers clockwise about 120°.

7 Lift off the upper part of the breathing system.



Place the upper part of the breathing system on the lower part.

8 Screw off the APL valve.



Align the APL valve correctly and tighten it with the knurled nut.

9 Remove the valves (red and blue).



Align the valves correctly. Insert the valves.



### Assembly

## Dräger Perseus® A500 Breathing System Reprocessing Instructions



	Manual cleaning and disinfection	Machine cleaning and disinfection	Sterilization
	✗	✓	✓
	✓	✗	✗
	✓	✗	✓
	✗	✓	✓

Position components so that all interior spaces and surfaces are completely flushed and water can drain off freely.

### Visual check



The following parts must be free of deposits:

- Incident flow grills
- Non-return valves (red and blue)

If required, remove deposits on the valve plates of the non-return valves with a soft cloth. If the incident flow grills are damaged, contact DrägerService®.

\* For machine cleaning, position each valve so that the valve cage is at the bottom.

This guideline does not replace the instruction for use.