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2 Cools F, Ofringa M, Askie LM. Elective high frequency oscillatory ventilation versus conventional ventilation for acute pulmonary dysfunction in preterm infants. Cochrane Database of Systematic Reviews 2015; 19;(3):CD000104.

3 Wilson-Costello D, Friedman H, Minich N, Fanaroff AA, Hack M. Improved survival rates with increased neurodevelopmental disability for extremely low birth weight infants in the 1990s. Pediatrics. 2005 Apr; 115(4):997-1003.

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4 Koltun M, Cattran CB, Kent G, Volgyesi G, Froese AB, Bryan AC. Oxygenation during high-frequency ventilation compared with conventional mechanical ventilation in two models of lung injury. *Anesth Analg*. 1982 Apr; 61(4): 323–332.

5 Rimensberger PC. ICU Cornerstone: High frequency ventilation is here to stay. *Crit Care* (2003) 7: 342.

6 González-Pacheco N, Sánchez-Luna M, Ramos-Navarro C, Navarro-Patiño N, de la Blanca ARS. Using very high frequencies with very low lung volumes during high-frequency oscillatory ventilation to protect the immature lung. A pilot study. *J Perinatol*. 2016 Apr; 36(4): 306–310.

7 Iscan B, Duman N, Tuzun F, et al. Impact of Volume Guarantee on High-Frequency Oscillatory Ventilation in Preterm Infants: A Randomized Crossover Clinical Trial. *Neonatology* 2015; 108(4):277-82.

8 Goldsmith JP, Karotkin EH, et al. Assisted Ventilation of the Neonate: An Evidence-Based Approach to Newborn Respiratory Care. Elsevier 6th Edition. Philadelphia 2017.

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Jane Pillow, Co-Director of the Centre of Neonatal Research and Education at the University of Western Australia

<sup>1</sup> Pillow J. High-Frequency Oscillatory Ventilation: Theory and Practical Applications. Drägerwerk AG & Co. KGaA. 2016: 9102693.

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