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www.draeger.com

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1. Hodgson LE, Murphy PB, Hart N. Respiratory management of the obese patient undergoing surgery. *J Thorac Dis.* 2015 May;7(5):943-52. [リンク](#)
2. Steier J, Jolley CJ, Seymour J, et al. Neural respiratory drive in obesity. *Thorax.* 2009 Aug;64(8):719-25. [リンク](#)
3. Pankow W, Podszus T, Gutheil T, et al. Expiratory flow limitation and intrinsic positive end-expiratory pressure in obesity. *J Appl Physiol.* 1998 Oct;85(4):1236-43. [リンク](#)
4. Salome CM, King GG, Berend N. Physiology of obesity and effects on lung function. *J Appl Physiol.* 2010 Jan;108(1):206-11. [リンク](#)
5. Eichenberger A, Proietti S, Wicky S, et al. Morbid obesity and postoperative pulmonary atelectasis: an underestimated problem. *Anesth Analg.* 2002 Dec;95(6):1788-92 [リンク](#)
6. Tanoubi I, Drolet P, Donati F. Optimizing preoxygenation in adults. *Can J Anaesth.* 2009 Jun;56(6):449-66. [リンク](#)
7. Edmark L, Östberg E, Scheer H, et al. Preserved oxygenation in obese patients receiving protective ventilation during laparoscopic surgery: a randomized controlled study. *Acta Anaesthesiol Scand.* 2016 Jan;60(1):26-35. [リンク](#)
8. Futier E, Constantin JM, Petit A, et al. Positive end-expiratory pressure improves end-expiratory lung volume but not oxygenation after induction of anaesthesia. *Eur J Anaesthesiol.* 2010 Jun;27(6):508-13. [リンク](#)
9. Sreejit MS, Ramkumar V. Effect of positive airway pressure during pre-oxygenation and induction of anaesthesia upon safe duration of apnoea. *Indian J Anaesth.* 2015 Apr;59(4):216-21. [リンク](#)
10. Bourouche G, Bourgain JL. Preoxygenation and general anaesthesia: a review. *Minerva Anesthesiol.* 2015 Aug;81(8):910-20 [リンク](#)
11. Murphy C, Wong DT. Airway management and oxygenation in obese patients. *Can J Anaesth.* 2013 Sep;60(9):929-45. [リンク](#)
12. Futier E, Constantin JM, Pelosi P, et al. Noninvasive ventilation and alveolar recruitment maneuver improve respiratory function during and after intubation of morbidly obese patients: a randomized controlled study. *Anesthesiology.* 2011 Jun;114(6):1354-63. [リンク](#)
13. Anästhesiologische Aspekte beim adipösen Patienten URL: <http://www.obesity-online.com/Anaesthesie.htm>; Viewed 14.05.2016
14. Brodsky JB. Positioning the morbidly obese patient for anaesthesia. *Obes Surg.* 2002 Dec;12(6):751-8. [リンク](#)
15. Rusca M, Proietti S, Schnyder P, et al. Prevention of atelectasis formation during induction of general anaesthesia. *Anesth Analg.* 2003 Dec;97(6):1835-9. [リンク](#)
16. Reinius H, Jonsson L, Gustafsson S, et al. Prevention of atelectasis in morbidly obese patients during general anaesthesia and paralysis: a computerized tomography study. *Anesthesiology.* 2009 Nov;111(5):979-87. [リンク](#)
17. Coussa M, Proietti S, Schnyder P, et al. Prevention of atelectasis formation during the induction of general anaesthesia in morbidly obese patients. *Anesth Analg.* 2004 May;98(5):1491-5 [リンク](#)
18. Deutsche Adipositas Gesellschaft, Die Prävalenz von Übergewicht und Adipositas in Deutschland, URL: <http://www.adipositas-gesellschaft.de/>; viewed 14.05.2016
19. Definition of pulmonary compliance; URL https://en.wikipedia.org/wiki/Pulmonary_compliance; viewed 14.05.2016
20. Dr. Chris Thompson, Senior Staff Specialist at Royal Prince Alfred Hospital, Clinical Lecturer at the University of Sydney, Australia, lecture at the ANZCA Congress 2015 in New Zealand, "Ventilation Techniques – A Practical Workshop"; URL: <https://www.youtube.com/watch?v=sU7sgTH2seA&feature=youtu.be>; viewed 14.05.2016 [リンク](#)
21. Benzing A., Pannan B. (Hrsg). *Praxishandbuch Anästhesie.* Deutscher Ärzteverlag, 2009, 2nd Edition, ISBN: 978-3-7691-1271-9
22. Aoragh, A.; Pastijn, E.; Cadiere, GB.; et al.. Rapid sequence induction versus routine induction in obese patients scheduled for bariatric surgery: 19AP3-6. *Eur J Anaesthesiol.* 2013 June;30:257-257 [リンク](#)
23. Neilipovitz DT, Crosby ET. No evidence for decreased incidence of aspiration after rapid sequence induction. *Can J Anaesth.* 2007 Sep;54(9):748-64. [リンク](#)

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Drägerwerk AG & Co. KGaA