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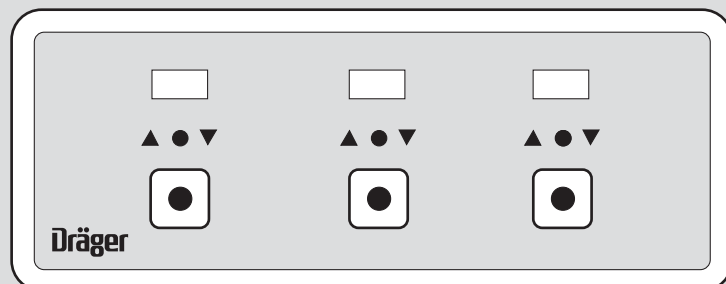
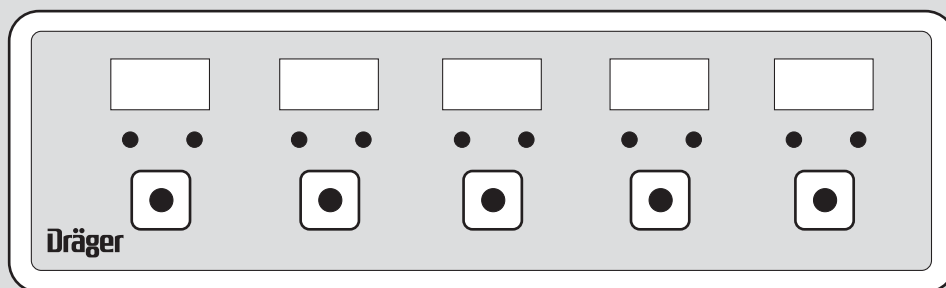
Drägerwerk AG & Co. KGaA

For Your Safety and that of Your Patients

Strictly follow the Instructions for Use

Any use of the apparatus requires full understanding and strict observation of these instructions. The apparatus is only to be used for purposes specified here.

Maintenance

The apparatus must be inspected and serviced regularly by trained service personnel at six monthly intervals (and a record kept). Repair and general overhaul of the apparatus may only be carried out by trained service personnel.

We recommend that a service contract be obtained with DrägerService and that all repairs also be carried out by them. Only authentic Dräger spare parts may be used for maintenance.

Observe chapter "Maintenance Intervals".

DANGER

Not for use in areas of explosion hazard.

This apparatus is neither approved nor certified for use in areas where combustible or explosive gas mixtures are likely to occur.

Liability for proper function or damage

Liability for the proper function of the apparatus is irrevocably transferred to the owner or operator to the extent that the apparatus is serviced or repaired by personnel not employed or authorized by DrägerService or if the apparatus is used in a manner not conforming to its intended use.

Dräger cannot be held responsible for damage caused by non-compliance with the recommendations given above.

The warranty and liability provisions of the terms of sale and delivery of Dräger are likewise not modified by the recommendations given above.

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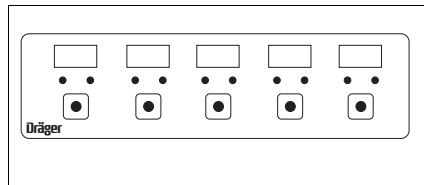
Intended Use

Operating- and emergency alarm panels

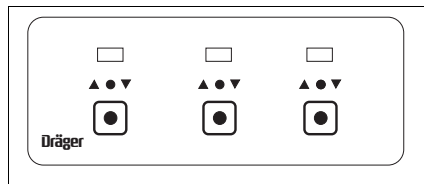
Display systems for operating conditions and activation of alarms, for use with medical gas pipeline systems.

Operating alarm panels:

Visual and audible alarms for the technical staff of the hospital, signalling certain operating conditions: e. g. empty cylinder manifold.

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Emergency alarm panels:

Visual and audible alarms of the medical staff in certain areas of the hospital e.g. surgical department or intensive-care ward, signalling failure of gas supply.

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Testing and Commissioning

The operating- and emergency signal panels are components of the medical gas supply system.

NOTE

The unit may not be operated until it has been tested and commissioned by trained and qualified personnel.

National regulations must be observed.

In the European Economic Area, EN 737-3.

When installation or maintenance measures have been carried out, a testing and commissioning program must be undertaken by trained staff.

These tests determine:

- whether the safety requirements for the protection of patients and staff have been fulfilled

and

- whether the performance characteristics of the medical gas pipeline system are met.

Written records of the tests must be kept.

Following this testing procedure, the pipeline system which is ready for operation and the documentation, are handed over to the user, and users are then given instruction.

The handing-over procedure must be recorded formally.

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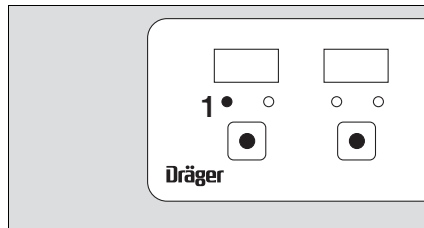
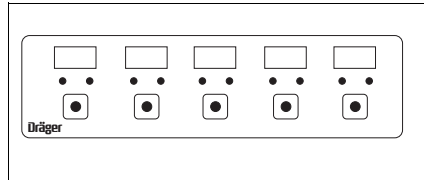
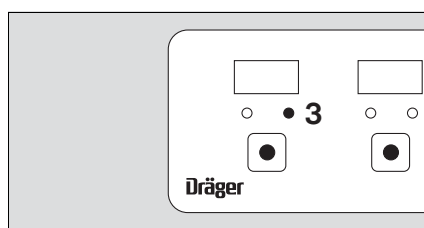
Operation

Operating alarm panels

These are controlled by contactors in the system to be monitored.

During operation,

when the system component monitored is ready for operation or in operation:

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the audio alarm and LED colour indicate the type of fault. Depending on the system installed, a red LED flashes (alarm) or a yellow LED flashes (warning).

Alarm:

- 2 Red LED for the gas concerned flashes,
- audio alarm sounds,
 - a potential-free relay is deactivated.
 - Rectify cause of fault immediately.

The cause of alarm may be e.g.:

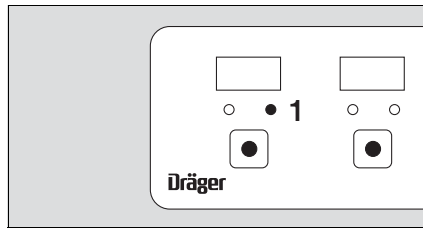
- operating pressure of gas in question incorrect,
- failure of liquid oxygen tank, compressor, refrigeration type air drier or vacuum pump,
- disconnection between contactor and operating alarm panel.

Warning (level 1):

- 3 Yellow LED for the gas concerned flashes,
- a potential-free relay is deactivated.

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
Warning (level 2):

- 1 Yellow LED for the gas concerned flashes,
 - audio alarm sounds,
 - a potential-free relay is deactivated.
 - Rectify cause of alarm. The cause of an alarm may be:
 - cylinder manifold on primary side of gas concerned is empty,
 - insufficient vessel pressure,
 - incorrect dew point – fault.

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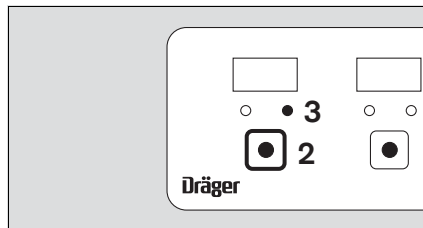The audio alarm can be suppressed:

- 2 Press pushbutton » ● « for the gas concerned. The audio alarm is muted,
- 3 Red/yellow LED changes from flashing to continuously lit.

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Unheeded alarms of short duration are displayed and should be acknowledged:

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If repeated short-duration alarms occur:

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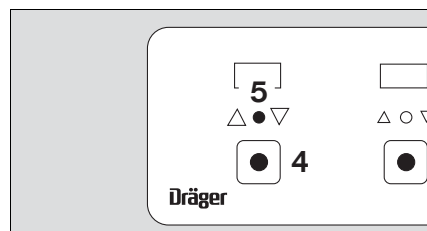
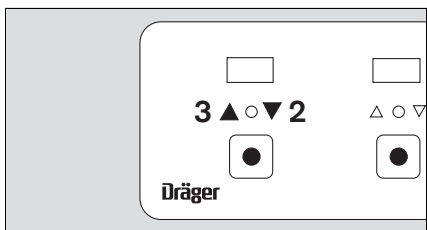
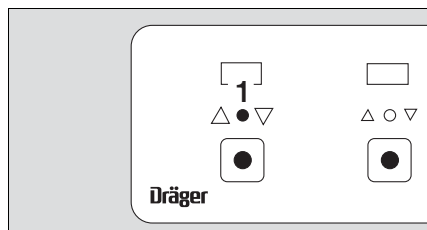
Emergency alarm panels

These are controlled by contactors installed downstream of the last shut-off valve in the valve boxes of individual areas.

During operation,

when the supply of individual gases is within prescribed values:

- 1 green LED of gas concerned is continuously lit.

In the case of malfunction or failure of gas supply:

- Audio alarm sounds,
- 2 red LED » ▼ « flashes if the pressure drops 20 % below the prescribed value or
- 3 red LED » ▲ « if the pressure rises 20 % above the prescribed value,
- a potential-free relay is deactivated.

The audio alarm can be suppressed for about 15 minutes:

- 4 Press » ● « pushbutton for the gas concerned.
- Audio alarm off,
- 5 red LED changes from flashing to lit continuously. The LED remains lit until the fault is rectified.
- Connect up an alternative gas supply as an emergency supply.

The cause of an alarm may be e.g.:

- pressure of gas concerned is above or below alarm limits,
- disconnection between contactor and emergency alarm panel.

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If fault is not rectified within about 15 minutes:

- audio alarm sounds again,
- red LED flashes again.

When the system has been appropriately installed, alarms are passed on as a malfunction message to a building management system (BMS).

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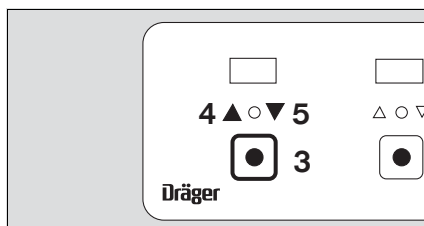
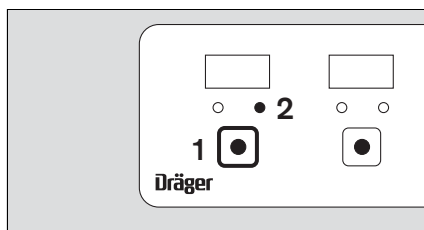
during operation

Operating alarm panel:

- 1 Hold down » ● « pushbutton,
- 2 LED remains lit as long as the pushbutton is held down.

Emergency alarm panel:

- 3 Hold down » ● « pushbutton,
- 4 » ▲ « LED and
- 5 » ▼ « LED remain lit as long as the pushbutton is held down.

If the operating or emergency alarm panel is connected with a building management system (BMS), the functional test is passed on like an alarm.

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Cleaning and Disinfecting

DANGER

Do not use cleaning agents which release chlorine or oxygen.

Do not use solvents such as benzene or ether.

- Wipe surface with a disposable cloth.

NOTE

Wipe disinfection only.

Follow disinfectant manufacturer's instructions.

Use surface disinfectants. For material compatibility use disinfectants based on

- aldehydes,
- alcohols,
- quaternary ammonium compounds.

Damage to materials may occur if disinfectants are used which are based on

- phenols,
- halogen-releasing compounds,
- strong organic acids,
- oxygen-releasing compounds.

For users in Germany we recommend that only disinfectants on the current DGHM list are used (DGHM: Deutsche Gesellschaft für Hygiene und Mikrobiologie = German Society for Hygiene and Microbiology).

The DGHM list indicates the composition of each disinfectant.

For countries where the DGHM list is not available, we recommend the types of disinfectant given above.

The following surface disinfectants can be recommended due to surface compatibility:

Disinfectant	Manufacturer
Buraton 10 F	Schülke & Mayr, Norderstedt
Kohrsolin	Bode Chemie GmbH&Co., Hamburg

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Ambient conditions

Temperature 0 to 40 °C
Relative humidity 20 to 95 %
Atmospheric pressure 800 to 1300 hPa

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Relay contacts

Contact load

potential-free two-way
contact
48 V AC/DC
1.0 A / 30 W / 60 VA

Elektromagnetická kompatibilita (EMC)

v souladu s normou
EN 60601-1-2

Electromagnetic Compatibility EMC

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18-046

UMDNS-Code

Universal Medical Device
Nomenclature System

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Třída I

Classification

according to EC Directive
93/42/EC Appendix IX

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FAX +49 451 8 82- 20 80

💻 <http://www.draeger.com>

Directive 93/42/EEC
concerning Medical Devices

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© Drägerwerk AG & Co. KGaA