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en – Supplement to the instructions for use for
Agila/Agila lift with the part no. 9039303
Movita/Movita lift with the part no. 9052693
Ponta C/H/E/S with the part no. 9039261
GeminaDUO with the part no. 9039783

This supplement must be kept with the instructions for use of the medical device.

- If several patient beds are connected to one Dräger ceiling supply unit (e.g., Ponta C/H/E/S), Dräger recommends to label the associated socket pairs for the IACS system cable.

A field for labeling the socket pairs is provided on the sockets.

WARNING

To properly use this medical device, read and comply the instructions for use and this supplement.

This supplement updates the information in the instructions for use of the medical device in the following chapters:

Getting started

Follow the instructions below to set up the IACS (Infinity Acute Care System) in combination with the IACS system cable of the Dräger ceiling supply unit:

CAUTION

Observe the instructions for use of the connected monitors C700 and M540.

- The IACS system cable must only be used in combination with the monitors C700 and M540 including the power supply unit (PS250/P2500).
- The C700 monitor must be directly connected to the power supply unit (PS250/P2500) using the IACS system cable. In this combination, the cable length of the IACS system cable must not exceed 4.5 m.

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© Dräger Medical GmbH
**Dräger Medical GmbH**
**Draeger Medical, Inc.**
U.S.A.
Drägerwerk AG & Co. KGaA