Fabius MRI

de	Ergänzung, Seite 2	hu	Kiegészítés, 17. oldal
en	Supplement, page 3	pl	Kiegészítés, strona 18
enUS	Supplement, page 4	tr	Ek, sayfa 19
fr	Supplément, page 5	el	Παράρτημα, σελίδα 20
es	Suplemento, página 6	ro	Supliment, pagina 21
it	Supplemento, pagina 7	ru	Дополнение, стр. 22
ptBR	Suplemento, página 8	zh	使用说明增补文档, 第 23 页
nl	Aanvulling, pagina 9	ko	부록, 24 참조
da	Tillæg, side 10	ja	補足文書, 25 ページ
no	Tillegg, side 11		
sv	Tillägg, sida 12		
cs	Dodatek, strana 13		
hr	Dodatak, str. 14		
sr	Dodatak, str. 15		
sk	Doplňok, strana 16		

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Supplement to the instructions for use

This supplement is used for the medical device indicated in the following table.

Medical device	Part number	Edition
Fabius MRI	9039036	Up to 8. Edition

- Enclose the supplement with the instructions for use of the medical device.

This supplement updates the information in the following chapter of the instructions for use.

Maintenance

The following precautionary statement has been added:

WARNING

Risk of fire

When replacing the battery, short-circuits or excessive temperatures can occur, resulting in fire or explosion. The battery must only be replaced by experts.

Preventive maintenance

The following precautionary statements have been added:

WARNING

Risk of faulty components

Device failures might result from wear and material fatigue of components.

To maintain the function of all components, this device must be inspected and serviced at the intervals specified by the manufacturer.

WARNING

Risk of electric shock

Before carrying out any maintenance work, disconnect all electrical connections from the power supply and gas connections from the gas supply.

The following interval has been added in the table for preventive maintenance:

Component	Interval	Measure	Responsible personnel
Internal lithium battery	Every 36 months	Replace	Experts

Repair

For repairs, Dräger recommends DrägerService and the use of original Dräger parts.

Supplement to the instructions for use

This supplement is used for the medical device indicated in the following table.

Medical device	Part number	Edition
Fabius MRI	9039035	Up to 10. Edition

- Enclose the supplement with the instructions for use of the medical device.

This supplement updates the information in the following chapter of the instructions for use.

Maintenance

The following precautionary statement has been added:

WARNING

Risk of fire

When replacing the battery, short-circuits or excessive temperatures can occur, resulting in fire or explosion. The battery must only be replaced by experts.

Preventive maintenance

The following precautionary statements have been added:

WARNING

Risk of faulty components

Device failures might result from wear and material fatigue of components. To maintain the function of all components, this device must be inspected and serviced at the intervals specified by the manufacturer.

WARNING

Risk of electric shock

Before carrying out any maintenance work, disconnect all electrical connections from the power supply and gas connections from the gas supply.

The following interval has been added in the table for preventive maintenance:

Component	Interval	Measure	Responsible personnel
Internal lithium battery	Every 36 months	Replace	Experts

Repair

For repairs, Dräger recommends DrägerService and the use of original Dräger parts.

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© Dräger Medical GmbH
**Dräger Medical GmbH**
Draeger Medical, Inc.
U.S.A.
Drägerwerk AG & Co. KGaA