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**Supplement to the Instructions for Use
Movita/Movita lift, order no. 90 37 852;
3rd edition – July 2006**

The supplement must be included with the medical device and replaces the previous information in the Instructions for Use of the medical device.

The following points differ from the existing Instructions for Use of the Movita/Movita lift:

Intended Use

Cable management (optional)

The cable management is used for mounting cables and hoses onto the Movita Column.

Preparation

Cable management

CAUTION!

Risk of damage to the device and personal injury!
Cable and hoses must not be kinked or twisted with excessive force.
Make sure no cables or hoses are damaged before placing them into the cable conduit.

CAUTION!

Risk of damage to the device and personal injury!
Only power cables with double insulation may be used.

- Press cables and hoses along the Movita Column through the flexible sealing lips into the cable conduit.
- Make sure to use the cables and hoses according to their instructions for use.

Care

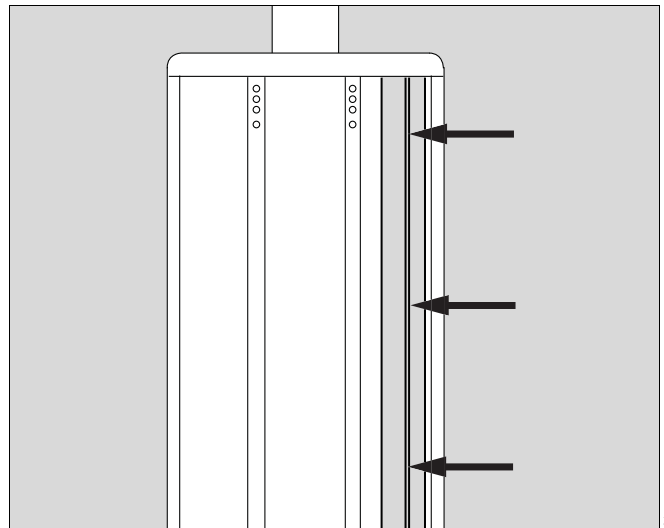
Cable management

- Remove cables and hoses from the cable conduit through the flexible sealing lips before cleaning.
- Wipe the inside and outside of the cable management using a damp cloth.
- Make sure that no liquid enters the interior of the cable conduit.
- Only if the interior of the cable conduit is completely dry, insert cables and hoses again.

Technical Data

Cable conduit capacity

- up to 4 gas hoses with a diameter of 14 mm
- up to 10 cables with a diameter of 7 mm

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© Dräger Medical GmbH
Dräger Medical GmbH
Drägerwerk AG & Co. KGaA