

Option CapnoPlus*/CO₂**

The CapnoPlus/CO₂ option provides CO₂ mainstream measurement for ventilated patients and is designed for use with Evita Infinity® V500, Evita V300, Evita XL, Evita 4 edition, Evita 2 dura und Savina 300



Your benefits with CO₂ mainstream measurement:

- quick and easy non-invasive CO₂ measurement¹⁾
- display of CO₂ values as end tidal CO₂ concentration and continuous CO₂ curves
- values such as CO₂ production (VCO₂), dead space (Vds) and the pulmonary dead space fraction (Vds/Vte) are also available


CO₂ AND CO₂ MEASUREMENT

Within the expiration phase, CO₂ is one of the most important parameters, providing valuable information on the efficacy of ventilation, gas exchange and metabolism.²⁾ Therefore, continuous monitoring of CO₂ concentrations can serve as an early warning system with regard to changes in the acuity level of critically ill patients.³⁾ The mainstream infrared absorption measurement of CO₂ directly at the Y-piece provides reliable data which are displayed in real time on the ventilator screen.

WHY CAPNOPLUS/CO₂?

CapnoPlus/CO₂ offers more than just CO₂ measurement.

- display capnometry and capnography simultaneously, together with other patient parameters such as flow, volume/pressure curves and data⁴⁾⁵⁾
- enhance patient safety with individually adjustable CO₂ alarm settings and optimized ventilation management
- obtain accurate measurements via the CapnoSmart sensor even under challenging conditions, such as during active circuit humidification
- avoid gas leakage and resulting misrepresentation of ventilation settings
- rapidly verify patient CO₂ values with direct sensor control via the reference filter
- perform zero calibration at the parc holder without disconnecting the patient from the ventilator
- combine CapnoPlus/CO₂ with optional mask ventilation (NIV/NIVplus) for real time CO₂ monitoring even during NIV
- take full advantage of the automated weaning protocol SmartCare®/PS for Evita Infinity V500, Evita V300 and Evita XL when you choose the CapnoPlus option

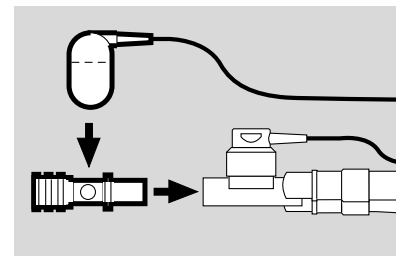

CO₂ measurement directly at the Y-piece

- 1) Bongard F, Sue D.: Pulse oximetry and capnography in intensive and transitional care units. West J. Med. 1992 Jan; 156(1); 57-64
- 2) St. John RE.: Exhaled gas analysis: technical and clinical aspects of capnography and oxygen consumption. Crit Care Nurs Clin N Am. 1989; 20:363-374
- 3) St. John RE.: End-tidal carbon dioxide monitoring Crit Care Nurs Vol 23, No. 4, August 2003; 83-88
- 4) AARC Guideline: Capnography/Capnometry during Mechanical Ventilation-2003 revision and update: Respiratory Care, May 2003 Vol. 48 No. 5
- 5) Behende et al.: Validity of a disposable and end-tidal CO₂ detection in verifying endotracheal tube placement in infants and children. Ann Erg Med 1992 31:142-5
- 6) Depending on ambient and ventilation pressure, the actual upper value, when indicated in Vol.%, can be lower than the upper value displayed.
- 7) STPD: Standard Temperature, Pressure, Dry. Measured values based on standardized physical conditions: 0 °C (32 °F), 1013 hPa, dry gas

* Evita XL, Evita 4 edition, Evita 2 dura

** Evita Infinity V500, Evita V300, Savina 300

CO₂ ACCESSORIES

– For the CapnoPlus option, Dräger offers reusable as well as disposable CO₂ cuvettes for adult and pediatric patients

– The patented design of the disposable CO₂ cuvette delivers the same performance as the reusable type, but without the additional cost of time consuming sterilization protocols.

TECHNICAL DATA**Option CapnoPlus/CO₂**

CO ₂ measurement in main flow	
End-expiratory CO ₂ concentration (etCO ₂)	Range 0 to 100 mmHg, or 0 to 15 Vol.% ⁶⁾ , or 0 to 13.3 kPa
CO ₂ production (VCO ₂)	Range 0 to 999 mL/min, STPD ⁷⁾
Monitoring	Capnometry and capnography

ORDER INFORMATION

CapnoPlus – option	order via Evita product configuration
CapnoPlus – retrofit kit	order no. 84 14 240
CO ₂ monitoring – option	order via Evita Infinity V500 product configuration
CO ₂ monitoring – retrofit kit	order no. 84 16 200
CO ₂ monitoring – option	order via Evita V300 product configuration
CO ₂ monitoring – retrofit kit	order no. 84 20 420
CO ₂ monitoring – option	order via Savina 300 product configuration
CO ₂ monitoring – retrofit kit	order no. 84 14 150



D-14954-2012

Mainstream CO₂ sensor for Evita Infinity V500, Evita V300, Savina 300



MT-0986-2007

Mainstream CO₂ sensor for Evita XL, Evita 4 edition, Evita 2 dura



D7089-2009

From left to right:
Disposable CO₂ cuvette, pediatric
Disposable CO₂ cuvette, adults
Reusable CO₂ cuvette, pediatric
Reusable CO₂ cuvette, adults

CORPORATE HEADQUARTERS

Drägerwerk AG & Co. KGaA
Moislinger Allee 53–55
23558 Lübeck, Germany

www.draeger.com

Manufacturer:

Drägerwerk AG & Co. KGaA
Moislinger Allee 53–55
23558 Lübeck, Germany

REGION EUROPE CENTRAL AND EUROPE NORTH

Drägerwerk AG & Co. KGaA
Moislinger Allee 53–55
23558 Lübeck, Germany
Tel +49 451 882 0
Fax +49 451 882 2080
info@draeger.com

REGION EUROPE SOUTH

Dräger Médical S.A.S.
Parc de Haute Technologie
d'Antony 2
25, rue Georges Besse
92182 Antony Cedex, France
Tel +33 1 46 11 56 00
Fax +33 1 40 96 97 20
d1mf-contact@draeger.com

REGION MIDDLE EAST, AFRICA

Drägerwerk AG & Co. KGaA
Branch Office
P.O. Box 505108
Dubai, United Arab Emirates
Tel +971 4 4294 600
Fax +971 4 4294 699
contactuae@draeger.com

REGION ASIA / PACIFIC

Draeger Medical
South East Asia Pte Ltd.
25 International Business Park
#04-27/29 German Centre
Singapore 609916, Singapore
Tel +65 6572 4388
Fax +65 6572 4399
asia.pacific@draeger.com

REGION CENTRAL AND SOUTH AMERICA

Dräger Panama Comercial
S. de R.L.
Complejo Business Park,
V tower, 10th floor
Panama City
Tel +507 377 9100
Fax +507 377 9130
contactcsa@draeger.com

Locate your Regional Sales Representative at:
www.draeger.com/contact

