Dodatak

Infinity[®] Acute Care System

UPOZORENJE

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Infinity[®] Acute Care System Softver VG7.1

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Biomed

System setup

The screenshot shows the 'Biomed' tab in the 'System setup' menu. The 'Allow external device reconnect' option is highlighted with a red box and a red arrow pointing to the text 'Dozvoli ponovno povezivanje spoljnog uređaja' below. Another red arrow points from the text 'Omogućeno' to the 'Enabled' button of the 'Allow external device reconnect' option. A vertical sidebar on the right contains various system setup options.

Option	Current State
Change clinical password	Key icon
Change biomedical password	Key icon
Simulation (basic)	On / Off
French NFC mode	On / Off
External display	Clone / Indep. display
Patient profile selection	On / Off
Adopt OR alarms	Auto / Manual
Adopt cardiac bypass	Auto / Manual
Allow external device reconnect	Enabled / Disabled
Restore factory settings	Restore factory settings

System Setup Options (Right Sidebar):

- Country
- Units of measure
- Patient monitor
- Name service
- Network setup
- Printer setup
- Recorder setup
- Service
- System integration
- IT setup

Dozvoli ponovno povezivanje spoljnog uređaja

Omogućeno

Obavljanje ponovnog povezivanja spoljnog uređaja

Nakon što omogućite **External device reconnect (Ponovno povezivanje spoljnog uređaja)** u meniju Biomed, obavite ponovno povezivanje spoljnog uređaja na sledeći način:

- 1 Kliknite na dugme **System setup... (Podešavanje sistema...)** na traci glavnog menija.

The screenshot shows the 'System setup' window with the 'Biomed' tab selected. The 'External device reconnect' option is highlighted with a red box and a line pointing to the 'Reconnect' button. The 'Reconnect' button is located at the bottom of the Biomed settings panel. The 'External device reconnect' option is described as: 'Communication with all external serial devices is temporarily lost during this process.'

Ponovno povezivanje

- 2 Odaberite karticu **Screen setup (Postavka ekrana)**.
- 3 Odaberite karticu **General settings (Opšte postavke)**.
- 4 Kliknite na dugme **Reconnect (Ponovno povezivanje)**.

Funkcija odvajanja na uređaju M540

Prikazaće se iskačući ekran **Confirm external device reconnect (Potvrdi ponovno povezivanje spoljnog uređaja)**:

Confirm external device reconnect (Potvrdi ponovno povezivanje spoljnog uređaja)

Connection to all external devices may be unavailable for up to three minutes.

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© Drägerwerk AG & Co. KG
Draeger Medical Systems, Inc.
U.S.A.
Drägerwerk AG & Co. KGaA