Perseus A500

de	Ergänzung, Seite 2	no	Tilleggsdokument, side 36
en	Supplement, page 4	pl	Suplement, strona 38
bg	допълнение, страница 6	pt	Documento suplementar, página 40
cs	Dodatek, strana 8	ptBR	Documento suplementar, página 42
da	Supplement, side 10	ro	Fișă de completare, pagina 44
el	Παράρτημα, σελίδα 12	ru	Приложение, стр. 46
es	Anexo, página 14	sk	Dodatok, strana 48
fi	Lisälehti, sivu 16	sl	Priloga, stran 50
fr	Complément, page 18	sr	Dodatak, strana 52
hr	Dodatni dokument, stranica 20	sv	Tilläggsinformation, sida 54
hu	Kiegészítő dokumentum, oldal 22	tr	Ek bilgi sayfası, sayfa 56
id	Tambahan, halaman 24	vi	Phần bổ sung, trang 58
it	Supplemento, pagina 26	zh	增补, 第 60 页
ja	補足説明書, ページ 28		
ko	부록, 페이지 30		
lt	Priedas, puslapis 32		
nl	Aanvullingsdocument, pagina 34		

WARNING

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Supplement to the instructions for use

Instructions for use	Part number	Edition
Perseus A500 SW 2.0n	9510595	3 – 2020-01 and higher

- Keep this supplement with the instructions for use.

This supplement updates the information of the instructions for use in the following chapters.

Assembly and preparation

Before first operation

Charging the battery

The following section has been changed:

WARNING

Risk due to reduced power supply from the internal battery

Batteries are wearing parts. The capacity of the battery diminishes with the period of use.

- **Avoid short charge and discharge cycles.**
- **When the device is switched on, do not disconnect it from the mains power supply.**
- **When the device is powered by the internal battery while switched on, ensure the functional integrity of the battery by performing a battery test at regular intervals. For further information see: "Inspection", page 5.**

Service

Inspection

The following section has been changed:

Measure	Interval	User group
Inspection and safety check	Every 12 months	Service personnel
Battery test ¹⁾	Every 3 months	Clinical user

1) If the device is connected to the mains power supply without interruption while switched on, the additional battery test is not necessary.

The following section has been added:

Battery test

Performing the battery test:

Prerequisite: The batteries must be fully charged.

- 1 Disconnect the device from the mains power supply. The **Power failure** alarm will be displayed.
- 2 Operate the device for 30 minutes in the volume-controlled ventilation mode. Use the following settings for this:
 - VT = 500 mL
 - RR = 10 /min
 - I:E = 1:1.5
 - PEEP = 5
 - FG flow = 10 L/min
 If the device does not display any of the **Battery failure**, **Battery charge low** or **Battery charge very low** alarms during these 30 minutes, the test has been passed.
- 3 Connect the mains power supply.

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© Drägerwerk AG & Co. KGaA
Drägerwerk AG & Co. KGaA