



Dräger Disposable Handle for Polaris Lights

Attaching the Disposable Handle

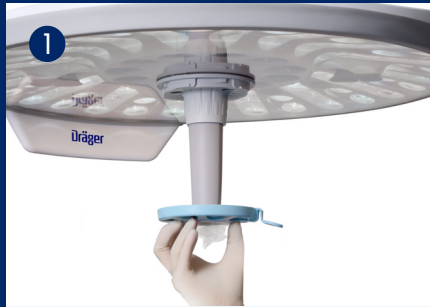

Remove sterile disposable handle from undamaged outer package. Apply to inner handle observing aseptic technique.

Note: Leave clear plastic sleeve in compressed state.

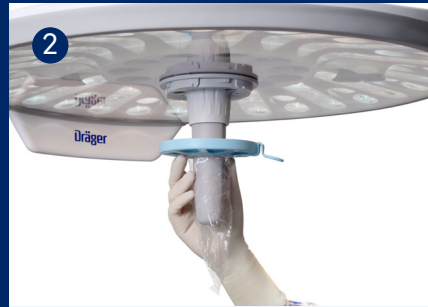

Hold the disposable handle by the recessed grips and fit it onto the inner handle. Avoid air pockets between the disposable handle and the inner handle.

Warning: The inner handle must be free from damage. If the inner handle is not free from damage, it may damage the disposable handle.



Turn the disposable handle clockwise until it is engaged to the light. The handle is now secured in place.

Warning: The disposable handle must be checked for secure attachment after it has been fitted to the inner handle. The disposable handle can fall down into the operating field if it is not engaged properly.

Removing the Disposable Handle



Using two hands, hold the fitted disposable handle in one hand, whilst pulling down on the breakaway ring-pull with the other.



Continue to pull down on the ring-pull until it breaks away from its original position.



Turn the recessed grips counter-clockwise, to completely remove the disposable handle. Dispose as per instructions protocol.

This guideline does not replace or amend the instructions for use. Please always refer to the instructions for use document for use of the disposable handle. The instructions for use and the hospital guidelines and policies must be observed at all times.