

WaterLock 2 Consumables and Accessories

Protects your patient, protects your gas measurement systems. Designed to give you reliable gas measurements, the Dräger WaterLock 2 helps you to effectively filter humid and contaminated exhaled air thanks to our advanced membrane technology. Keeping your patients and investments safe from water, bacteria, and potential viruses.



Benefits

Deliver safe and precise gas measurement

When connecting patients to your anaesthesia or gas measurement device, you need to be reassured that your measurements are accurate. Moreover, controlling the likelihood of cross-contaminants is essential for your patient's wellbeing.

Engineered with two PTFE-membranes in combination with two self-sealing filters, our WaterLock 2 effectively separates condensed water and secretions that contain harmful contaminants, such as bacteria and viruses. As second protection system the WaterLock 2 comes with super absorbent polymer seals behind the PTFE membranes. In case of a fluid breakthrough these seals block the gas flow and protect the measuring unit against water (and contaminant) ingress — giving you more precise gas measurements as well as safeguarding your patient's health.

Safe and simple to handle

Maximise your workflow efficiency and improve your infection control protocols. Simply connect and operate, WaterLock 2's easy snap-lock connection makes it easy for you to install and empty. Using a 3-step process, all you need to do is:

- Hold WaterLock 2 at the grip and insert into your device until it audibly clicks
- Connect gas sample line to Luer port
- Extract water with a syringe

You can reliably use WaterLock for up to four weeks in your unit.

In combination with state-of-the-art PTFE membranes, you can effectively prevent water, bacteria, or viruses from entering your gas measurement system — keeping infections at bay.

Stable and secure: How WaterLock 2 works

Fitted with two hydrophobic microporous PTFE filters, WaterLock 2 provides your patient with optimum protection during therapy while ensuring accurate gas readings. This is achieved through a dual system that utilises self-sealing filters as second protection level. To begin, a sample of gas exhaled by the patient passes through the main hydrophobic membrane which filters both water and contaminants before it passes on through the module's sensors. The water is drawn into the water tank which can be emptied using a standard syringe. Optimised to ensure you a fast response and extremely effective drainage, our Dräger WaterLock 2 maintains a perfect capnogram even under extreme conditions, e.g. minimal-flow anaesthesia.

Reducing your costs

Your medical equipment is a valuable investment that helps you improve your patient outcomes and save lives. To support you in managing your costs of care, Dräger's WaterLock 2 has an operating life of four weeks, with immediate replacement, if necessary, when outstanding conditions occur. During this period, it can be emptied and reused as often as necessary, thanks to the possibility to remove water directly by using a standard syringe.

Benefits

Ordering Information

	Order no.
WaterLock 2, 12 pcs.	6872130
Infinity ID WaterLock 2, 12 pcs.	6872020
Sample line, disposable, 10 pcs.	8290286

Not all products, features, or services are for sale in all countries.
Mentioned Trademarks are only registered in certain countries and not necessarily in the country in which this material is released. Go to www.draeger.com/trademarks to find the current status.

CORPORATE HEADQUARTERS
Drägerwerk AG & Co. KGaA
Moislinger Allee 53–55
23558 Lübeck, Germany
www.draeger.com

Manufacturer:
Drägerwerk AG & Co. KGaA
Moislinger Allee 53-55
23542 Lübeck, Germany

REGION DACH
Drägerwerk AG & Co. KGaA
Moislinger Allee 53–55
23558 Lübeck, Germany
Tel +49 451 882 0
Fax +49 451 882 2080
info@draeger.com

REGION EUROPE
Drägerwerk AG & Co. KGaA
Moislinger Allee 53–55
23558 Lübeck, Germany
Tel +49 451 882 0
Fax +49 451 882 2080
info@draeger.com

REGION MIDDLE EAST, AFRICA
Drägerwerk AG & Co. KGaA
Branch Office
P.O. Box 505108
Dubai, United Arab Emirates
Tel +971 4 4294 600
Fax +971 4 4294 699
contactuae@draeger.com

REGION ASIA PACIFIC
Dräger Singapore Pte. Ltd.
61 Science Park Road
The Galen #04-01
Singapore 117525
Tel: +65 6872 9288
Fax: +65 6259 0398
asia.pacific@draeger.com

REGION NORTH AMERICA
Dräger, Inc.
3135 Quarry Road
Telford, PA 18969, USA
Tel +1 800 4DRAGER
(+1 800 437 2437)
Fax +1 215 723 5935
info.usa@draeger.com

REGION CENTRAL AND SOUTH AMERICA
Dräger Indústria e Comércio Ltda.
Al. Pucurui - 51 - Tamboré
06460-100 - Barueri - São Paulo
Tel. +55 (11) 4689-4900
relacionamento@draeger.com

Locate your Regional Sales
Representative at:
www.draeger.com/contact

