

## Dräger X-plore® 1900

### Donning procedure



D-8184-2018

91 06 879 | 19.01-1 | HQ | PP | Subject to modifications | © 2019 Drägerwerk AG & Co. KGaA



D-16667-2018

Hold the mask in one hand with the head straps hanging down. Expand the loop on the harness until the top loop almost lies against the half mask.



D-16684-2018

Unfold the half mask and slightly preform the centre of the nose clip.



D-16704-2018

Hold the half mask under the chin and over the nose. Stretch the bottom loop and pull it over the head to the neck.



D-16712-2018

Position the top loop high on the back of the head.



D-16723-2018

Adjust the length of the loops so that the half mask sits comfortably on the head and the tension is evenly distributed.



D-16729-2018

Adjust the nose clip to the shape of the nose, using both hands.



D-16733-2018

To check the seal, cover the half mask with both hands and, if necessary, seal the exhalation valve. Exhale forcefully.



D-16743-2018 / D-16747-2018

If air leaks into the nose area, readjust the nose clip. If air leaks at the edges, adjust the half mask and check the fit of the harness on the head.



D-16756-2018

Ready for use.

Always read and understand the instructions for use prior to use!