Zeus / Zeus *Infinity* Empowered

de	Ergänzung zu den Gebrauchsanweisungen, Seite 2	hr	Dopuna Upute za uporabu, stranica 13
en	Supplement to the Instructions for Use, page 3	sr	Dopuna uputstva za korišćenje, strana 14
fr	Supplément au manuel d'utilisation, page 4	hu	Kiegészítés a használati útmutatók- hoz, 15. oldal
es	Suplemento para las instrucciones de uso, página 5	pl	Załącznik do instrukcji obsługi, strona 16
it	Supplemento alle istruzioni per l'uso, pagina 6	tr	Kullanma Kılavuzuna Ek, Sayfa 17
ptIB	Suplemento das Instruções de Utilização, página 7	el	Παράρτημα των οδηγιών χρήσης, σελίδα 18
ptBR	Suplemento às instruções de uso, página 8	ro	Fișă de completare la instrucțiunile de utilizare, pagina 19
nl	Aanvulling op de gebruiksaanwijzing, pagina 9	ru	Дополнение к руководству по эксплуатации, стр. 20
sv	Bilaga till Bruksanvisning, sida 10	zh	使用说明书的补充说明, 第 21 页
fi	Lisälehti käyttöohjeeseen, sivu 11	ko	사용 지침서 부록, 페이지 22
cs	Doplňěk k návodu k použití, strana 12		

WARNING

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Supplement to the Instructions for Use

Zeus Anesthesia Workstation SW 3.n with the Order No. 9038192; 2nd Edition

Zeus Anesthesia Workstation SW 4.n with the Order No. 9039205; 2nd Edition and higher

Zeus *Infinity Empowered* Anesthesia Workstation (Zeus *IE*) SW 1.n with the Order No. 9039982; 1st Edition and higher

Zeus *Infinity Empowered* Anesthesia Workstation (Zeus *IE*) SW 2.n with the Order No. 9054908; 1st Edition

Zeus Retrofit Kit C500 Anesthesia Workstation System (Zeus C500) SW 1.n with the Order No. 9052817; 1st Edition and higher

and

Zeus Retrofit Kit C500 Anesthesia Workstation System (Zeus C500) SW 2.n with the Order No. 9055542; 1st Edition

This supplement must be attached to the medical device. It updates the information in the Instructions for Use of the medical device.

The following points differ from the existing Instructions for Use of the Zeus Anesthesia Workstation and replace the previous information in the Instructions for Use:

Instructions for Use Zeus SW 3.n

Care and Maintenance

Maintenance intervals

DIVA	After 2 years.
Metering module for desflurane	Maintenance is free of charge at the factory service center in Lübeck. A module on loan is provided during maintenance (Please inform DrägerService in advance).

Instructions for Use Zeus SW 4.n

Care and Maintenance

Maintenance intervals

Replacement by trained personnel

DIVA	After 2 years.
Metering module for desflurane	Maintenance is free of charge at the factory service center in Lübeck. A module on loan is provided during maintenance (Please inform DrägerService in advance).

Instructions for Use Zeus *IE* SW 1.n

Zeus *IE* SW 2.n

Zeus C500 SW 1.n

Zeus C500 SW 2.n

Maintenance

Maintenance intervals for Zeus *IE*

Replacement by trained personnel

DIVA	After 2 years.
Metering module for desflurane	Maintenance is free of charge at the factory service center in Lübeck. A module on loan is provided during maintenance (Please inform DrägerService in advance).

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© Drägerwerk AG & Co. KGaA
Drägerwerk AG & Co. KGaA