

## Supplement to instructions for use

---

### WARNING

**To properly use this medical device, read and comply with the instructions for use and this supplement.**

This supplement applies to instructions for use issued before April 2015.

- Keep this supplement with the instructions for use.

This supplement updates the information of the instructions for use in the chapter "Cleaning, disinfection, and sterilization".

### Testing of procedures and agents

The surface disinfectants listed in the instructions for use, for which material compatibility and effectiveness have been verified, can continue to be used.

### Manual disinfection and simultaneous cleaning

The indicated surface disinfectants and recommendations regarding classes of active ingredients are replaced by those in the following list:

Class of active ingredient	Product name	Manufacturer
Chlorine-releasing agents	Actichlor <sup>®</sup> plus	Ecolab
	BruTab 6S <sup>®</sup>	Brulin
	Clorox Professional Disinfecting Bleach Cleaner	Clorox
	Dispatch <sup>®</sup> Hospital Cleaner Disinfectant Towels with Bleach	
	Klorsept <sup>®</sup> 17	Medentech

Class of active ingredient	Product name	Manufacturer
Oxygen-releasing agents	Descogen <sup>®</sup> Liquid	Antiseptica
	Descogen <sup>®</sup> Liquid r.f.u.	
	Dismozon <sup>®</sup> plus	Bode Chemie
	Dismozon <sup>®</sup> pur	
	Oxycide <sup>®</sup>	Ecolab USA
	Perform <sup>®</sup>	Schülke & Mayr
	Virkon <sup>®</sup>	DuPont
Quaternary ammonium compounds	Mikrozid <sup>®</sup> sensitive liquid <sup>1)</sup>	Schülke & Mayr
	Mikrozid <sup>®</sup> alcohol free liquid <sup>1)</sup>	
	Mikrozid <sup>®</sup> sensitive wipes <sup>1)</sup>	
	Mikrozid <sup>®</sup> alcohol free wipes <sup>1)</sup>	
	acryl-des <sup>®1)</sup>	
Aldehydes	Buraton <sup>®</sup> 10 F	

1) virucidal against enveloped viruses

Dräger recommends using a surface disinfectant from the list above. Other disinfectants are used at own risk.

Dräger points out that oxygen- and chlorine-releasing agents may cause color change in some materials. This is not an indication that the product is not functioning correctly.

At the time of validation, the disinfectants listed above showed material compatibility.

The manufacturers of the surface disinfectants have verified at least the following spectra of activity:





- Bactericidal
- Yeastcidal
- Virucidal or virucidal against enveloped viruses

Observe the specifications of the surface disinfectant manufacturers.

Use surface disinfectants that are nationally authorized.

#### Manufacturer

##### **Drägerwerk AG & Co. KGaA**

 Moisinger Allee 53 – 55  
D-23542 Lübeck  
Germany  
 +49 451 8 82-0  
FAX  +49 451 8 82-20 80  
 <http://www.draeger.com>

9054867 – EB en

© Drägerwerk AG & Co. KGaA

Edition: 2 – 2015-07

(Edition: 1 – 2014-12)

Dräger reserves the right to make modifications to the device without prior notice.

