

## Reprocessing the Dräger Caleo



**!** Please reprocess the device according to your hospital's guidelines. This document is for information purposes only. Please refer to the device instructions for use for complete disassembly, assembly, cleaning and surface disinfection.

**Requirement:** There must be no patient in the device.

### Surface disinfection with cleaning for:

- Basic device/trolley
- All disassembled parts
- Hood and side walls (interior and exterior)
- X-ray drawer

### Machine cleaning and thermal disinfection:

- Grommets
- Fan impeller
- Water tank (option)

**Before starting use, ensure that all surfaces are dry and free from disinfectant residues.**

### Disassembly ►

**1** Set the incubator to the lowest position.



**18** Set the incubator to the desired working height.

**2** Close and remove the transfer set.



**17** Connect and open the transfer set.

**3** Remove sterile water bag or water tank (option).



**16** Connect water tank or sterile water bag.

**4** Remove upper double wall and continuous seal.



**15** Apply continuous seal and refit upper double wall.

**5** Remove hood.



**14** Refit hood.

**6** Cleaning mode



**13**

### ◀ Assembly

**!** Only use disinfectants that have received national approval and that are suitable for the reprocessing procedure in question. Ensure that no liquid enters the device.

## Disassembly ▶

<p>7 Open side walls.</p>	<p>8 Remove grommets.</p>	<p>9 Open head and foot access ports.</p>	<p>10 Remove mattress.</p>
			
<p>12 Close side walls.</p>	<p>11 Insert grommets.</p>	<p>10 Close head and foot access ports.</p>	<p>9 Insert mattress.</p>
<p>11 Remove mattress tray.</p>	<p>12 Remove hose holder arm (option).</p>	<p>13 Unlock and remove x-ray tray.</p>	<p>14 Remove trough.</p>
			
<p>8 Insert mattress tray.</p>	<p>7 Insert hose holder arm (option).</p>	<p>6 Insert and lock x-ray tray.</p>	<p>5 Insert trough.</p>
<p>15 Remove fan impeller and connecting piece.</p>	<p>16 Remove drawer.</p>	<p>17 Clean and disinfect basic device in full.</p>	<p>18 Remove display unit rotary knob and clean and disinfect display unit.</p>
			
<p>4 Insert fan impeller and connecting piece.</p>	<p>3 Insert drawer.</p>	<p>2</p>	<p>1 Insert display unit rotary knob.</p>

◀ Assembly