

## Dräger Fabius Family Compact Breathing System Disassembly/Assembly



### Disassembly

**1** Remove all hoses, cables and the absorber, lift off the breathing system.

**9** Mount the breathing system, connect all hoses, cables and the absorber.

**2** Unscrew CLIC adapter (optional) or absorber canister. Inspect the CLIC adapter seal.

**8** Make sure all seals are in place. Screw on CLIC adapter (optional) or absorber canister.

**3** Insert cleaning disc into the adapter. Close adapter until it snaps.

**7** Remove cleaning disc from the adapter. Close adapter until it snaps.

**4** Loosen and pull out expiratory port; pull out flow sensor. Remove flow-sensor guard.

**6** Attach flow-sensor guard. Insert flow sensor, insert and tighten expiratory port.

**5** Remove inspiratory and expiratory valve.

**5** Mount inspiratory and expiratory valve.

**6** Remove APL-valve.

**4** Mount APL-valve.

**7** Remove hook for gas sample line (optional).

**3** Mount hook for gas sample line (optional).

**8** Remove waste-gas port.

**2** Mount waste-gas port.

**9** Remove cover of ventilator chamber, remove ventilator diaphragm.

**1** Insert ventilator diaphragm and mount cover of ventilator chamber.

### Assembly

## Dräger Fabius Family Compact Breathing System Reprocessing Instructions



	Manual cleaning and disinfection	Machine cleaning and disinfection	Sterilization
	✘	✔	✔
	✔	✘	✘
	✔	✘	✔
	✘	✔	✘

Position components so that all interior spaces and surfaces are completely flushed and water can drain off freely.

### Visual check



After the machine cleaning and disinfection, a hot steam sterilization is required to dry the breathing system. Valves, ventilator cover, and AGS sleeve: Make sure that the cleaning and rinsing liquids can flow in the opening direction of the valve. Remove any water which may have accumulated in the ventilator diaphragm. For further instructions and explanations please see the respective instructions for use.

This guideline does not replace the instruction for use.