

## Dräger Zeus® Breathing System Disassembly/Assembly



### Disassembly



<p><b>1</b> Unscrew the CLIC adapter (optional).</p>  <p>Screw the CLIC adapter (optional). <b>8</b></p>	<p><b>2</b> Insert cleaning disc into the adapter. Close adapter until it snaps.</p>  <p>Remove cleaning disc from the adapter. Close adapter until it snaps. <b>7</b></p>	<p><b>3</b> Loosen and pull out the expiration port.</p>  <p>Check the expiration grills.* Insert and tighten the expiration port. <b>6</b></p>
<p><b>4</b> Loosen and pull out the inspiration port.</p>  <p>Check the inspiration grills. Insert and tighten the inspiration port. <b>5</b></p>	<p><b>5</b> Remove the flow sensors.</p>  <p>Insert the flow sensors <b>4</b></p>	<p><b>6</b> Turn the screws using the key counterclockwise about 120°.</p>  <p>Turn the screws using the key clockwise about 120°. <b>3</b></p>
<p><b>7</b> Lift off the upper part of the breathing system.</p>  <p>Set the upper part on the lower part of the breathing system. <b>2</b></p>	<p><b>8</b> Screw off the APL valve.</p>  <p>Screw in the APL valve. <b>1</b></p>	

\* In case the grill is damaged contact DrägerService®

### Assembly

## Dräger Zeus® Breathing System Reprocessing Instructions



	Manual cleaning and disinfection	Machine cleaning and disinfection	Sterilization
	✘	✔	✔
	✔	✘	✔

Position components so that all interior spaces and surfaces are completely flushed and water can drain off freely.

### Visual check



#### Manual cleaning and disinfection:

Please use the disinfectants listed in the Instructions for use of the Dräger flow sensors.

#### Machine cleaning and disinfection:

Recommendation: Position components so that all interior spaces and surfaces are completely flushed and water can drain off freely.

#### Sterilization:

SpiroLife® flow sensors are not suitable for plasma or radiation sterilization.

This guideline does not replace the instruction for use.