

# DID YOU KNOW?




## QuickSet

### What is it?

With QuickSet is possible to modify a parameter and have an immediate effect, without any need of confirmation. The QuickSet is allowing to have a realtime adjustment of the parameter with a realtime monitoring on the screen of the ventilator

### How to use it?

1. Select the parameter
2. Button then turns yellow
3. Hold down rotary knob for 1,5 seconds
4. Button turns green with yellow rim
5. Turn the knob while held down
6. Settings are put in place immediately

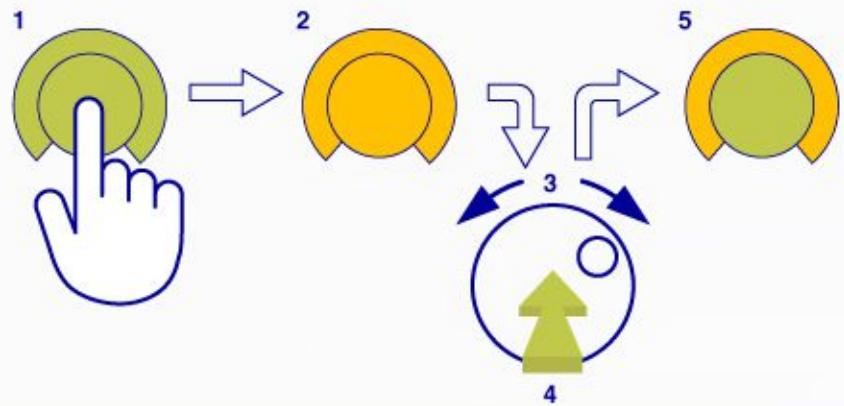


### Why is it relevant?

In some recruitment maneuvers techniques is important to have a continuous adjustment in real time of the parameters, without losing time in confirming and/or waiting for the parameter to change.

### Why is it helpful to improve outcomes?

QuickSet is helpful to perform some recruitment maneuvers, PEEP leaving inspiratory pressure at the same level<sup>1</sup>

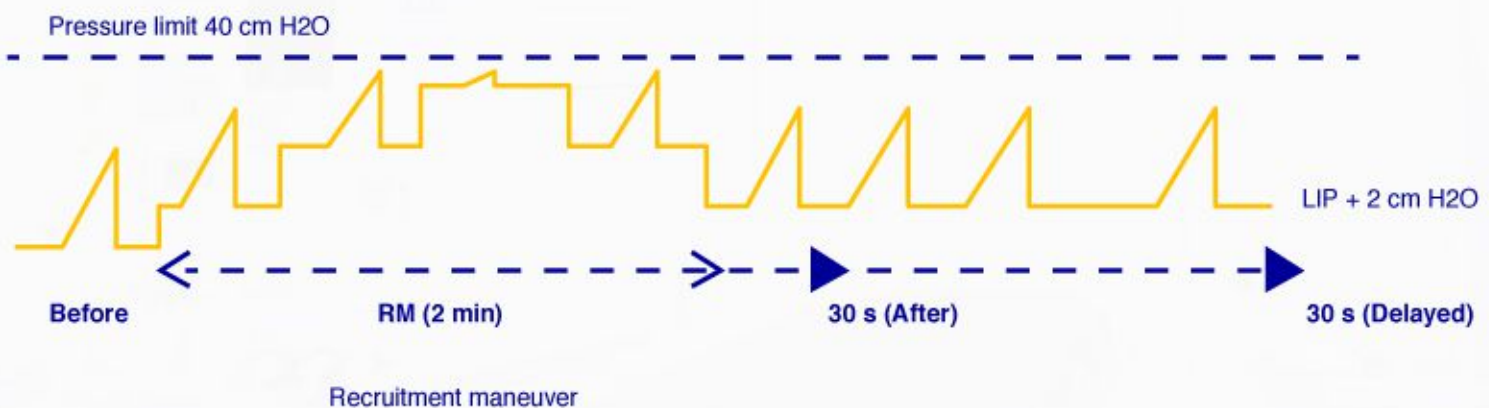

1. Lim et al., Morphometric Effects of the Recruitment Maneuver on Saline-lavaged Canine Lungs: A Computed Tomographic Analysis.