- Agila/Agila lift**
(Sachnummer 90 37 845)
- Agila/Agila lift/Agila EasyLift**
(Sachnummer 90 39 302)
- Movita/Movita lift**
(Sachnummer 90 37 852)
- Forta lift**
(Sachnummer 90 37 859)
- Gemina**
(Sachnummer 90 29 048)
- Ponta C/H/E/S**
(Sachnummer 90 39 153)

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(en) Supplement (order no. 90 39 703) for the Instructions for Use

Agila/Agila lift
(order no. 90 37 845)

Agila/Agila lift/Agila EasyLift
(order no. 90 39 303)

Movita/Movita lift
(order no. 90 37 852)

Forta lift
(order no. 90 37 859)

Gemina
(order no. 90 29 048)

Ponta C/H/E/S
(order no. 90 39 261)

CAUTION!

After replacing the pressure hoses for medical gas supply, the following checks must be carried out according to ISO 7396-1:

- leak test,
- clogging test (flow test) and
- gas type check.

After replacing the hoses of the anesthesia gas scavenging system (AGSS), the following checks must be carried out according to EN 737-2:

- leak test,
- clogging test (flow test) and
- pressure drop test.

The following points were added to the "Maintenance" chapter:

Intervals of maintenance and replacement

WARNING!

The medical device must be inspected and serviced regularly by trained service personnel. Repair of the medical device may also only be carried out by trained service personnel. After each maintenance, a proper check of the system must be carried out. Dräger recommends that a service contract be obtained with DrägerService and that all repairs also be carried out by them. Dräger recommends that only authentic Dräger repair parts be used for maintenance. Otherwise the correct functioning of the medical device may be compromised. Observe chapter "Maintenance".

Functional inspection; visual inspection and leak test of all pressure hoses.	First inspection 5 years after initial operation, then annual inspections. Inspection by trained experts.
Visual inspection of all cables and checking of electrical safety.	
Observe the respective manufacturer's Instructions for Use for terminal units.	Every 6 years by trained experts.
Replacement of the wear parts, i.e. O-ring and flat sealing ring in Dräger terminal units.	
General overhaul and replacement of pressure hoses and cables.	Every 12 years by trained experts.

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© Dräger Medical GmbH
🏠 Dräger Medical GmbH
Drägerwerk AG & Co. KGaA