Fabius family

de	Ergänzung, Seite 2	sl	Priloga, stran 42
en	Supplement, page 4	bg	Допълнение, Страница 44
fr	Complément, page 6	el	Παράρτημα, σελίδα 46
es	Anexo, página 8	tr	Ek bilgi sayfası, sayfa 48
it	Foglio informativo, pagina 10	zh	增补, 第 50 页
ptBR	Documento suplementar, página 12	ko	부록, 페이지 52
nl	Aanvullingsdocument, pagina 14	ja	補足説明書, ページ 54
da	Indlægsseddel, side 16		
no	Tilleggsdokument, side 18		
sv	Tilläggsinformation, sida 20		
lt	Papildymas, puslapis 22		
lv	Papildu dokumenti, lapp. 24		
ru	Приложение, стр. 26		
pl	Suplement, strona 28		
cs	Dodatek, strana 30		
sk	Dodatok, strana 32		
hu	Kiegészítő dokumentum, oldal 34		
hr	Dodatni dokument, stranica 36		
ro	Fișă de completare, pagina 38		
sr	Dodatak, strana 40		

WARNING

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Fabius family

Supplement to the instructions for use

Device	Part number	Edition
Fabius Tiro	9038686	Up to 7. Edition
Fabius GS <i>premium</i>	9038923	Up to 7. Edition
Fabius MRI	9039036	Up to 8. Edition
Fabius <i>plus</i>	9039054	Up to 4. Edition
Fabius <i>plus</i> XL	9052882	Up to 1. Edition

- Keep this supplement with the instructions for use of the medical device.

The supplement updates the information of the instructions for use in the following chapters:

For your safety and that of your patients

Product-specific safety information

The following warning statements are added:

WARNING

Risk of fire

The flow sensor can ignite medications or other substances based on highly flammable substances.

- Do not nebulize medications or other substances that are easily flammable or spray them into the device.
- Do not use substances containing alcohol.
- Do not allow flammable or explosive substances to enter the breathing system or the breathing circuit.
- Do not use cyclopropane or ether.

WARNING

Risk due to failure of flow measurement

Deposits that were not removed during reprocessing can damage the measuring wires in the flow sensor or cause a fire.

- Before inserting the flow sensor check for visible damage, soiling, and particles. Repeat this check regularly.
- Replace flow sensors when damaged, soiled, or not particle-free.

Assembly and preparation

Preparation

Inserting the flow sensor

The following warning statement is added:

WARNING

Risk of fire

Residual vapors of easily flammable disinfectants (e.g., alcohols) and deposits that were not removed during reprocessing can ignite when the flow sensor is in use.

- **Ensure particle-free cleaning and disinfection.**
- **After disinfection, allow the flow sensor to air for at least 30 minutes.**
- **Before inserting the flow sensor, check for visible damage and soiling such as residual mucus, medication aerosols, and particles.**
- **Replace flow sensors when damaged, soiled, or not particle-free.**

Cleaning, disinfection and sterilization

Reprocessing methods

Visual inspection

The following warning statements are added:

WARNING

Risk of fire

Residual vapors of easily flammable disinfectants (e.g., alcohols) and deposits that were not removed during reprocessing can ignite when the flow sensor is in use.

- **Ensure particle-free cleaning and disinfection.**
- **After disinfection, allow the flow sensor to air for at least 30 minutes.**
- **Before inserting the flow sensor, check for visible damage and soiling such as residual mucus, medication aerosols, and particles.**
- **Replace flow sensors when damaged, soiled, or not particle-free.**

CAUTION

Risk of failure of flow measurement

Improper reprocessing and soiling, such as deposits or particles, can damage the flow sensor.

- No machine cleaning or disinfection
- No plasma sterilization or radiation sterilization
- No water jets, compressed air, brushes or the like
- No ultrasonic bath
- No hot-steam sterilization with Spirolog and Infinity ID flow sensors
- Clean and disinfect the flow sensor in accordance with the corresponding instructions for use.
- For disinfecting the flow sensor use only clean disinfectant solutions.

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© Dräger Medical GmbH
Dräger Medical GmbH
Drägerwerk AG & Co. KGaA