Fabius

de	Ergänzung, Seite 2	sl	Dopolnilo èlanku, str. 32
en	Supplement, page 4	hu	Kiegészítés, 34. oldal
enUS	Supplement, page 6	pl	Dodatek, strona 36
fr	Supplément, page 8	tr	Ek, sayfa 38
es	Suplemento, página 10	el	Παράρτημα, σελίδα 40
it	Supplemento, pagina 12	lt	Papildymas, 42 psl.
pt_br	Suplemento, página 14	lv	Papildinājums, 44. lpp.
nl	Aanvulling, pagina 16	ro	Supliment, pagina 46
da	Tillæg, side 18	ru	Дополнение, стр. 48
no	Tillegg, side 20	bg	Приложение, стр. 50
sv	Tillägg, sida 22	zh	使用说明增补文档, 第 52 页
cs	Dodatek, strana 24	ko	부록, 54 참조
hr	Dodatak, str. 26	ja	補足文書, 56 ページ
sr	Dodatak, str. 28		
sk	Doplňok, strana 30		

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Supplement to the instructions for use

This supplement is for use with the medical devices listed in the table.

Medical device	Part number	Edition
Fabius GS <i>premium</i>	9038923	1st edition and higher
Fabius Tiro	9038686	1st edition and higher
Fabius plus	9039054	1st edition and higher
Fabius plus XL	9052882	1st edition and higher
Fabius MRI	9039036	1st edition and higher
Fabius CE	9037441	1st edition and higher
Fabius GS	9038685	1st edition and higher

- Keep this supplement with the instructions for use of the medical device.

NOTE

Chapter and section headings in older editions of instructions for use may deviate from those used in this supplement.

The supplement updates the information of the instructions for use in the following chapters:

Maintenance

The following interval was added to the table indicating intervals for preventive maintenance:

Component	Interval	Task	Responsible personnel
Pressure reducers for gas cylinders with pin index connections ¹⁾	After 6 years	Exchange	Experts

1) optional

Appendix – Daily and Pre-use Checkout Form

Gas supply

- Visually inspect the central gas supply and the gas cylinders. Connect all hoses. Make sure that the hoses are tightly connected.
- Make sure that the supply pressures of the central gas supply lie within the permissible range.

Gas cylinders with pin-index connection

- Open the gas cylinders (if present).
- O₂ pressure is higher than 1000 psi (70 kPa x 100).
- N₂O pressure is higher than 600 psi (43 kPa x 100) if present.
- Air pressure is higher than 1000 psi (70 kPa x 100) if present.
- Close the gas cylinders.
- Observe the pressure gauge.
- The displayed value is constant.
- To release the excess pressure, open the flow control valves.
- Close the flow control valves.

Supplement to the instructions for use

This supplement is for use with the medical devices listed in the table.

Medical device	Part number	Edition
Fabius GS <i>premium</i>	9038906	1st edition and higher
Fabius Tiro	9038723	1st edition and higher
Fabius Tiro M	9039710	1st edition and higher
Fabius OS	9052124	1st edition and higher
Fabius MRI	9039035	1st edition and higher
Fabius GS	9038705	1st edition and higher

- Keep this supplement with the instructions for use of the medical device.

NOTE

Chapter and section headings in older editions of instructions for use may deviate from those used in this supplement.

The supplement updates the information of the instructions for use in the following chapters:

Maintenance

The following interval was added to the table indicating intervals for preventive maintenance:

Component	Interval	Task	Responsible personnel
Pressure reducers for gas cylinders with pin index connections ¹⁾	After 6 years	Exchange	Experts

1) optional

Appendix – Daily and Pre-use Checkout Form

Gas supply

- Visually inspect the central gas supply and the gas cylinders. Connect all hoses. Make sure that the hoses are tightly connected.
- Make sure that the supply pressures of the central gas supply lie within the permissible range.

Gas cylinders with pin-index connection

- Open the gas cylinders (if present).
- O₂ pressure is higher than 1000 psi (70 kPa x 100).
- N₂O pressure is higher than 600 psi (43 kPa x 100) if present.
- Air pressure is higher than 1000 psi (70 kPa x 100) if present.
- Close the gas cylinders.
- Observe the pressure gauge.
- The displayed value is constant.
- To release the excess pressure, open the flow control valves.
- Close the flow control valves.

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© Dräger Medical GmbH
Dräger Medical GmbH
Drägerwerk AG & Co. KGaA