en  3

fi  4

no  5

sv  6

## Alcotest<sup>®</sup> 4000 / 5000 / 6000 / 7000 Safety-related information

Download

[www.draeger.com/ifu](http://www.draeger.com/ifu)

## 1 Safety information for Alcotest<sup>®</sup> 4000 / 5000 / 6000 / 7000

### WARNING

Small parts of the instrument can come off and be swallowed. The instrument is not intended for children under the age of 3. Explosion hazard! Do not take the instrument anywhere near flames and do not open it using force.

### CAUTION

Damage to the instrument: Do not charge alkaline batteries as this could otherwise damage the instrument. Do not immerse the instrument in any liquids. Do not use any cleaning agents which contain solvents or limonene. Any battery acid escaping can damage the instrument. Remove batteries if the instrument is not being used for a long period of time.

### NOTICE (for use with a funnel)

Avoid sampling in a draught/wind in rooms or open areas. It could falsify the test result. Keep it hygienic: Do not touch the top part of the funnel. It is essential to keep a distance of 2 – 3 cm from the test person to be tested. Do not touch the skin of a test person and secretions or surfaces with the funnel. Avoid direct contact with substances to be tested. Replace a contaminated funnel.

### DISPOSAL

This product must not be disposed of as household waste. Dispose of batteries and rechargeable batteries at battery collection centres as specified by the applicable regulations. You can return this product to Dräger free of charge. For information please contact the national marketing organizations or Dräger.

### Ambient conditions

When operating the Alcotest<sup>®</sup> 4000 / 6000: -5 °C to 50 °C (23 °F to +122 °F)

When operating the Alcotest<sup>®</sup> 5000: -15 °C to +50 °C (+5 °F to +122 °F)

When operating the Alcotest<sup>®</sup> 7000: -10 °C to 55 °C (14 °F to +131 °F)

When operating all instruments: 10 to 100 % RH (non condensing),  
600 to 1300 hPa

During storage: -20 °C to +60 °C (-4 °F to +140 °F), 15 to 75 % RH  
(10 to 100 % RH <48 h)

### Miscellaneous

USB port: on Alcotest<sup>®</sup> 4000 / 5000 / 6000 only for Service

CE marking: Electromagnetic compatibility

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© Dräger Safety AG & Co. KGaA
**Dräger Safety AG & Co. KGaA**