

Dräger DrugTest 5000 Test Kit

The Dräger DrugTest 5000 system consists of two main components: the DrugTest 5000 Test Kits and the DrugTest 5000 Analyzer. It is a fast and highly accurate system for testing saliva samples for substances such as amphetamines, designer amphetamines, opiates, cocaine and its metabolites, benzodiazepines, cannabinoids, ketamine or methadone.



The DrugTest 5000 Test Kits are ergonomically designed, easy to handle and immediately ready for use. They consist of a test cassette and a sample collector. The test kit can be handled with or without gloves by right-handed or left-handed persons. The test result is clearly displayed, making misinterpretation impossible. Made from a rigid porous material with capillary properties, this non-invasive test allows hygienic handling. Any contact with the sample and its components is prevented.

The test is performed in five simple steps, excluding incorrect operation.

The test itself is very straightforward: The operator removes the protective cap from the sample collector and hands the test cassette over to the subject, who puts the collector into the mouth and moves it back and forth between the cheek and the gum to extract a saliva sample.

When the collector has absorbed enough liquid, the integrated indicator will turn blue and the test cassette can be returned to the operator. The operator, who supervises the entire sampling process, then inserts the test cassette and the cartridge into the Analyzer. After a few minutes, the result will be clearly displayed. In order to facilitate matching of samples to subjects, the test cassette can be labelled with the subject's name and the date.

If required, the Dräger DrugTest 5000 can be used in conjunction with the saliva sample collector Dräger DCD 5000. This allows taking a second sample as well as storing and transporting the samples for laboratory analysis. If used in combination with the Dräger SSK 5000, it takes only a few minutes to determine whether a suspicious surface, preparation or substance may be associated with one of the designated types of narcotics.



D-31774-2011

The Dräger DrugTest 5000 Test Kit:
easy to use, fast, reliable and secure in testing for drugs

ORDERING INFORMATION

Designation

Designation	Order no.
Each Dräger DrugTest 5000 test kit consists of a sample collector test cassette and a cartridge, packaged in a foil pouch.	
Dräger DrugTest 5000 Training Test Kit	83 23 640
Dräger DrugTest 5000 Test Kit COC-OPI-BZO-THC-AMP-MET*	83 19 830
Dräger DrugTest 5000 Test Kit COC-OPI-THC-AMP-MET*	83 23 439
Dräger DrugTest 5000 Test Kit COC-OPI*	83 19 990
Dräger DrugTest 5000 Test Kit COC-OPI-BZO-THC-AMP-MET-MTD*	83 23 156
Dräger DrugTest 5000 Test Kit COC-OPI-BZO-THC-AMP-MET-MTD-KET*	83 23 646**
Packaging: 20 complete test kits per package	

*COC: cocaine – *OPI: opiates – *BZO: benzodiazepines – *THC: tetrahydrocannabinol (cannabis) – *AMP: amphetamine – *MET: methamphetamine + designer amphetamines –


*MTD: methadone – *KET: ketamine

**Only sold within the European Union


Accessories

Dräger DCD 5000 Saliva Sample Collector	83 19 910
Dräger SSK 5000 Surface Test Kit	83 20 490


DRÄGER DRUGTEST 5000 – THE TEST IS PERFORMED IN 5 SIMPLE STEPS:

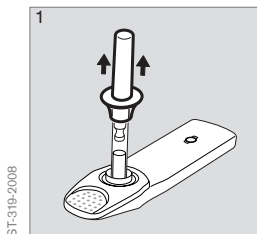


The protective cap is removed

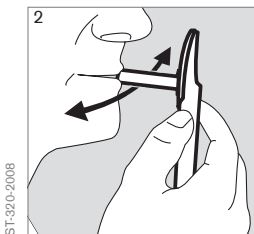


The sample collector is inserted into the mouth

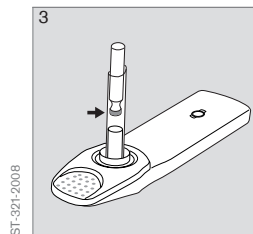

The integrated indicator turns blue

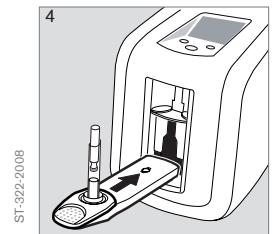

Test cassette and cartridge are inserted into the Analyzer

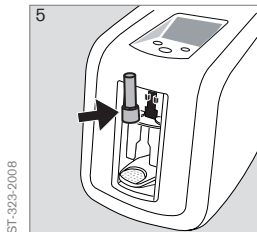

The display shows a clear result

Storage and shelf life

Test kits must be stored at temperatures between 4 and 30 °C (39 to 86 °F) in their original foil pouch packaging. Do not use a test kit if the enveloping foil pouch is broken and has, for example, holes or tears. Immediately use the test kit after opening the packaging. The test kit must not be used after the expiration date, which is clearly printed on the foil pouch.

CORPORATE HEADQUARTERS

Drägerwerk AG & Co. KGaA
Moislinger Allee 53–55
23558 Lübeck, Germany

www.draeger.com

REGION EUROPE

Dräger Safety AG & Co. KGaA
Revalstraße 1
23560 Lübeck, Germany
Tel +49 451 882 0
Fax +49 451 882 2080
info@draeger.com

REGION MIDDLE EAST, AFRICA

Dräger Safety AG & Co. KGaA
Branch Office
P.O. Box 505108
Dubai, United Arab Emirates
Tel +971 4 4294 600
Fax +971 4 4294 699
contactuae@draeger.com

REGION ASIA PACIFIC

Draeger Singapore Pte. Ltd.
61 Science Park Road
The Galen #04-01
Singapore 117525
Tel +65 6872 9288
Fax +65 6259 0398
asia.pacific@draeger.com

REGION CENTRAL AND SOUTH AMERICA

Dräger Indústria e Comércio Ltda.
Al. Pucurui - 51 - Tamboré
06460-100 - Barueri - São Paulo
Tel. +55 (11) 4689-4900
relacionamento@draeger.com

Locate your Regional Sales
Representative at:
www.draeger.com/contact

