

Dräger



D-9763-2011

Dräger DrugTest® System

The new standard in oral
fluid screening

Dräger. Technology for Life®

D-9764-2011



Accurate, on-the-spot oral fluid drug detection

TWO STEP SYSTEM

In the past, roadside drug screening has been difficult because it involved the collection of urine or blood samples. The Dräger DrugTest 5000 mobile oral fluid screening system provides an alternative.



D-54723-2012



D-31773-2011

Now, roadside drug screening can be performed with ease and results are provided within minutes. This non-invasive method lets you screen for seven types of the most commonly abused drugs—

while eliminating the embarrassment and hassle of collecting urine or blood samples.

System components include:

- Dräger DrugTest 5000 Test Kit: a ready-to-use test cassette with a built-in sample collector and volume adequacy indicator.
- Dräger DrugTest 5000 Analyzer: a rechargeable, fully automated analyzer providing controlled accurate analysis on site.



For Law Enforcement Use Only



D-9765-2011

EASY EVALUATION

The DrugTest 5000 system is designed for ease of use and accurate results.

- Quick and simple to operate
- User-friendly design
- On-screen instructions guide you through the test

NON-INVASIVE

Sample collection is completely non-invasive—which is easier for both you and the donor. When the cassette is handled correctly, you will not come into contact with the donor's oral fluid.

QUICK CONFIRMATION

The device controls analysis time and internal temperature for accurate results. Positive results can be confirmed with a second oral fluid sample via an independent, third-party laboratory analysis. Results are available online in a matter of days.

STORES, MANAGES AND PRINTS RESULTS

The analyzer stores the last 500 test results, along with the date and time. You can enter additional subject or test details using the compact keyboard. Data can be downloaded to a PC or printed using the Dräger Mobile Printer.

DRUG ANALYTICAL CUT-OFFS

Dräger DrugTest® 5000 Test Kit 7 Panel Cut-Off Levels

AMP D-amphetamine	50 ng/mL
MAMP D-methamphetamine	35 ng/mL (MDMA 75)
COC Cocaine	20 ng/mL
OPI Morphine	20 ng/mL
THC D9-tetrahydrocannabinol	5 ng/mL
BENZO Diazepam	15 ng/mL
MTD Methadone	20 ng/mL

EVALUATES UNKNOWN SUBSTANCES

You can also evaluate an unknown substance right on the spot. Simply collect a sample with the Dräger Surface Sampling Kit (SSK) 5000, transfer it onto the collection tip of the test kit, then analyze the sample for immediate results.




Dräger SSK 5000




DrugTest® 5000 Test Kit

PERFORM A TEST

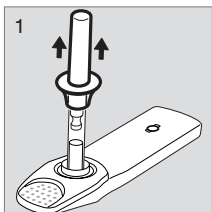


ST-319-2008

Remove the protective cap from the oral fluid collection tip.

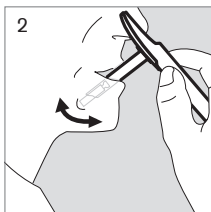


D-4780-2011

Insert the collection tip into the oral cavity, repeatedly swiping the surface of the cheeks and tongue.

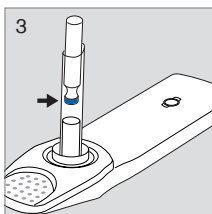


ST-319-2008

The indicator tip will turn blue when a sufficient sample volume has been collected. Failure of the indicator to turn blue does not invalidate the test.

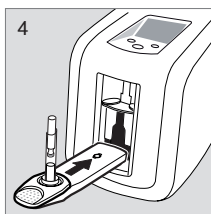


ST-322-2008

Insert the cassette into the analyzer.

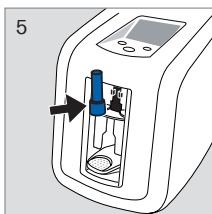

ST-319-2008

Insert the buffer cartridge into the analyzer and close the door. The analysis will proceed automatically.

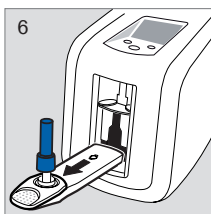

ST-15164-2008

All entered data and results can be printed via the Dräger Mobile Printer. Remove the cassette and attached cartridge from the device.

Not all products, features, or services are for sale in all countries. Mentioned Trademarks are only registered in certain countries and not necessarily in the country in which this material is released. Go to www.draeger.com/trademarks to find the current status.

CORPORATE HEADQUARTERS

Drägerwerk AG & Co. KGaA
Moislinger Allee 53–55
23558 Lübeck, Germany

www.draeger.com

Customer Service:

USA

+1 800-4DRAGER
(+1 800-437-2437)

Technical Service:

USA

+1 800-4DRAGER
(+1 800-437-2437)

Locate your Regional Sales
Representative at:

www.draeger.com/contact

