


Resuscitaire® Neonatal Open Care

The Resuscitaire® is the ideal device to have during labour and delivery procedures. It combines an effective warming therapy platform along with the components you need for clinical emergency and resuscitation.

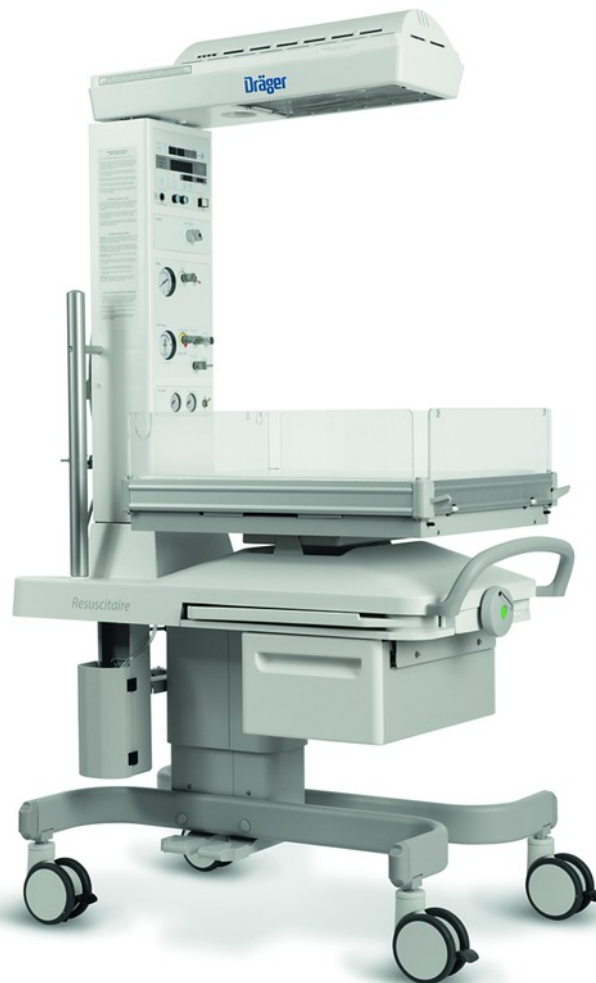

Benefits

Resuscitation right where you need it

The blender, suction, flowmeter, and positive pressure are all mounted on the bed and accessible at your finger tips.

AutoBreath™

The T-Piece resuscitation is the new standard for neonatal resuscitation. The Autobreath option allows you to perform this resuscitation hands free. You only have to set the breath per minute, FIO₂, PEEP, and airway pressure relief. All these parameters are adjustable based on the clinical needs of the infant and situation.

Silica Quartz Heater

The Silica Quartz technology, a fast heating element, provides complete and EVEN heating. A parabolic reflector helps distribute the heat evenly over the entire mattress area.

Alarm Free Pre Warm Mode

Pre warm mode allows caregivers to focus on the mother and baby without having to divert attention to silence alarms.

Technical Data

DRÄGER RESUSCITAIRE® WARMER

Physical Attributes (without options/accessories)		VHA Model
Height	72 in (183 cm)	71-79 in (180-201 cm)
Width	26.5 in (68 cm)	26.5 in (68 cm)
Depth	44 in (112 cm)	44 in (112 cm)
Weight (without options /accessories)	220-280 lbs (100-127 kg)	220-280 lbs (100-127 kg)
Mattress		
Height from floor	39.4 in (100 cm)	35-43 in (90-109 cm)
Width	21.0 in (53.3 cm)	21.0 in (53.3 cm)
Length	26.0 in (66.0 cm)	26.0 in (66.0 cm)
Thickness	1.0 in (2.54 cm)	1.0 in (2.54 cm)
Maximum capacity	15 lbs (4.54 kg)	15 lbs (4.54 kg)
Maximum Power Requirements		
Model	120V	120V, 50-60Hz, 750 W
	220/240V	220-240V, 50-60Hz, 750 W
Examination light		
Illumination		>0.11 lumens/cm (100 ft. Candles)
Quartz bulb type		50 W (1,076 lux)
Controller heating alarms		
Check patient		15 minutes in Manual Mode
Baby temperature		+/- 1° C from Set Point
High temperature		Skin Temp. 39.0° C +/- 0.2° C
Probe		Short or open circuit / No probe
System fail		Indicates System Fail
Power fail		AC Power Interruption
Alarm silence/reset intervals		
Check patient		Resets clock for 15 minutes manual mode
Baby temperature		10 minutes
High temperature		2 minutes
Procedural silence		Presilences baby temp alarm for 5 minutes
Set temperature/heater output		
Set temp range		34-38° C (>37.0° override)
Heater output		0-100%, 10% increments
Skin temp display		18-43° C
Accuracy		+/- 0.2° C (31° C to 37° C)
Resuscitation module		
Wall supply pressure		40-75 psi (275-517 kPa)
Cylinder pressure		2,900 psi max (19,994 kPa)
Patient gas supply		
Flow control range		0-15 LPM
Airway pressure limit-Adjustable		0-50 cm H ₂ O (0-4.9 kPa)
Internal preset limit		60 cm H ₂ O (5.9 kPa)

Technical Data

Auxiliary flow circuit

Auxiliary flow range	0-15 LPM
Fixed internal	160 cm H ₂ O (15.7 kPa)

Suction circuit

Adjustable	0-150 mmHg (0-20 kPa)
Disposable collection container	50 or 800 ml

Autobreath infant resuscitator

Adjustable breath rate range	18-60 BPM (+/-10% of setting)
I:E ratio	Fixed at 1:2 Nominal
Pressure (PEEP)	0-18 H ₂ O (0-1.77 kPa)
Gas bleed	5 LPM Max
Precision blender (optional)	21-100% O ₂ +/-3% O ₂

Alerts

Manual mode	System alerts every 30 seconds > 10 minutes, for 15 minutes
Apgar timer	Alerts at 1, 5 and 10 minutes
Timer	Visual timer up to 1 hour

Instrument tray

Length	13.0 in (33.0 cm)
Width	9.0 in (22.8 cm)
Capacity	5 lbs (2.2 kg)

X-ray cassette tray

Pocket width	14.5 in (36.8 cm)
Pocket length	11.0 in (27.9 cm)
Thickness	0.75 in (1.9 cm)

IV pole

Maximum load	5 lbs (2.2 kg)
--------------	----------------

Monitor shelf

Length	12.0 in (31.75 cm)
Width	14.0 in (40.64 cm)
Maximum load	10 lbs (4.5 kg)

DRÄGER RESUSCITAIRE® WARMER – WALL MOUNT

Maximum Power Requirements

Model	120V	120 v, 50-60 Hz, 750 W
	220/240V	220-240 v, 50-60 Hz, 750 W

Examination light

Illumination	> 100 Ft. Candles (0.11 Lumens/cm)
--------------	-------------------------------------

Specifications

Length	29.8 in (75.7 cm)
Width	10 in (25.4 cm)
Height	6 in (15.2 cm)
Weight	<35 lbs (15.92 kg)

Controller heating alarms

Check patient	5 minutes in Manual Mode (alerts every 30 seconds > 10 minutes)
Baby temp	±1° C from Set Point

Technical Data

High temp	Skin Temp 39.0° C ±0.2° C
Probe	Short or open circuit/No probe
System fail	Indicates System Fail
Power fail	AC Power Interruption
Alarm	Silence/Reset Intervals (manual mode)
Check patient	Resets clock for 15 minutes
Baby temp	10 minutes
High temp	2 minutes
Procedural silence	Silences audible Baby Temp alarm for 5 minutes
Set temperature/heater output	
Set temp range	34-38° C (.37.0° C override)
Heater output	0-100%, 10% increments
Skin temp display	18-43° C
Accuracy	±0.2° C (31° C to 37° C)

Notes

CORPORATE HEADQUARTERS

Drägerwerk AG & Co. KGaA
Moislinger Allee 53–55
23558 Lübeck, Germany
www.draeger.com

Manufacturer:

Dräger Medical Systems, Inc.
3135 Quarry Road
Telford, PA 18969-1042, USA

REGION EUROPE CENTRAL AND EUROPE NORTH

Drägerwerk AG & Co. KGaA
Moislinger Allee 53-55
23558 Lübeck, Germany
Tel +49 451 882 0
Fax +49 451 882 2080
info@draeger.com

REGION EUROPE SOUTH

Dräger Médical S.A.S.
Parc de Haute Technologie
d'Antony 2
25, rue Georges Besse
92182 Antony Cedex, France
Tel +33 1 46 11 56 00
Fax +33 1 40 96 97 20
d1mfr-contact@draeger.com

REGION MIDDLE EAST, AFRICA

Drägerwerk AG & Co. KGaA
Branch Office
P.O. Box 505108
Dubai, United Arab Emirates
Tel +971 4 4294 600
Fax +971 4 4294 699
contactuae@draeger.com

REGION ASIA / PACIFIC

Dräger Medical
South East Asia Pte Ltd.
25 International Business Park
#04-27/29 German Centre
Singapore 609916, Singapore
Tel +65 6572 4388
Fax +65 6572 4399
asia.pacific@draeger.com

REGION CENTRAL AND SOUTH AMERICA

Dräger Panama Comercial
S. de R.L.
Complejo Business Park,
V tower, 10th floor
Panama City
Tel +507 377 9100
Fax +507 377 9130
contactcsa@draeger.com

Locate your Regional
Sales Representative at:
www.draeger.com/contact

