

## Dräger Savina® 300 Select ICU Ventilation and Respiratory Monitoring

The Dräger Savina® 300 Select (in this configuration) combines the independence and power of a turbine-driven ventilation system with sophisticated ventilation modes. Its broad variety of features and accessories supports a patient range from babies\* to adults. The large colour touch screen and intuitive operating system that concentrates on established high class features make configuration and operation very simple.



D-14953-2019

\* starting from 5 kg bodyweight

## Benefits

---

### Ease-of-use

- Intuitive for simple operation and quick configuration
  - Dräger-wide standardised user interface provides confidence in use and reduces training time
  - Quick operational readiness with an automatic device check
  - Safe and fast initial start of ventilation thanks to pre-set start configuration based on patient height or patient category
  - Intelligent alarm handling for a quick response to patient alarm situations
  - Smooth and sealed surfaces for easy cleaning and disinfection
- 

### High ventilation performance

- Sophisticated ventilation modes for critically ill patients (e.g. PC-APRV, VC-MMV, AutoFlow®)
  - Enhanced trigger detection and precise ventilation settings allow the ventilation of babies starting from 5 kg bodyweight
  - Automatic Tube Compensation ATC® reduces the work of breathing for intubated patients with spontaneous breathing<sup>1</sup>
  - Stress-free spontaneous breathing with excellent trigger response time thanks to the turbine
  - Free breathing with AutoFlow in volume constant ventilation at a minimum pressure level
  - Non-invasive ventilation (NIV) available in all modes with a very quick response times to patient efforts
  - No device change needed in case of altered ventilation therapy: O<sub>2</sub>-therapy allows oxygen application with constant flow
  - Extended graphic capabilities with loops, trends and logbook
- 

### Independent from gas and power supply

- Built-in-turbine with rapid response time and continuous high flow delivery of up to 250 L/min
  - Five hours of independent ventilation due to built-in and external batteries
  - Transport Supply Unit (TSU) can be quickly attached for ergonomic handling of gas cylinders
  - Bed coupling for quick connection between ventilator and patient bed
  - Low Pressure Oxygen (LPO) inlet for ventilation without central gas supply
- 

<sup>1</sup> Respiratory comfort of automatic tube compensation and inspiratory pressure support in conscious humans  
Guttman, J. et al, Intensive Care Medicine 1997, Vol. 23, No.11, 1119-1124

## Related Products

D-3602-2019



---

### Dräger Evita® V800

Experience the next level of ventilator operation. The Evita® V800 combines high performance ventilation with an aesthetic design enabling quick and efficient operation. From the first onset of a lung protective ventilation until the integration of a patient care-centred intensive care workplace.

D-12050-2019



---

### Dräger Evita® V600

Experience the next level of ventilator operation. The Evita® V600 combines high performance ventilation with an aesthetic design enabling quick and efficient operation. From the first onset of a lung protective ventilation until the integration of a patient care-centred intensive care workplace.

## Technical Data

### Ventilation modes

Volume-controlled ventilation modes	<ul style="list-style-type: none"> <li>- VC-CMV / VC-AC</li> <li>- VC-SIMV</li> <li>- VC-MMV</li> </ul>
Pressure-controlled ventilation modes	<ul style="list-style-type: none"> <li>- PC-APRV</li> <li>- PC-BIPAP<sup>1</sup> / PC-SIMV+</li> <li>- PC-AC</li> </ul>
Support of spontaneous breathing	<ul style="list-style-type: none"> <li>- SPN-CPAP</li> </ul>

### Enhancements

	<ul style="list-style-type: none"> <li>- AutoFlow® – Automatic adaption of the inspiratory flow in volume orientated ventilation modes.</li> <li>- NIV – Non Invasive Ventilation with optimised alarm systems and automatic leakage compensation.</li> <li>- Paediatric Plus – allowing the ventilation of babies starting from 5 kg bodyweight</li> <li>- Capnography – Mainstream CO<sub>2</sub> measurement</li> <li>- MonitoringPlus – Loops, Trends, user Logbook</li> <li>- LPO – Low Pressure Oxygen. Independent oxygen supply, e.g. with an O<sub>2</sub> concentrator</li> <li>- Nurse call – Connection for transmitting alarm signals to a central, alarm system</li> <li>- Automatic tube compensation ATC® – Automatic tube compensation regulates the airway pressure to the tracheal level</li> <li>- O<sub>2</sub>-therapy – continuous flow is applied via an oxygen mask, a hood or nasal cannula for patients with independent breathing</li> </ul>
--	--

Patient type	Adult, paediatric patients, and babies starting from 5 kg bodyweight
Respiratory rate	2/min to 80/min
Inspiration time	0.2 to 10 s
Tidal volume	0.05 to 2.0 L, BTPS <sup>2</sup> with PaediatricPlus 0.02 to 2.0 L, BTPS
Inspiratory pressure	1 to 99 mbar (or hPa or cmH <sub>2</sub> O) (1 mbar = 100 Pa)
PEEP/interm. PEEP	0 to 50 mbar (or hPa or cmH <sub>2</sub> O)
Pressure support/ $\Delta P_{supp}$	0 to 50 mbar (or hPa or cmH <sub>2</sub> O) (relative to PEEP)
Flow acceleration	5 to 200 mbar/s (or hPa/s or cmH <sub>2</sub> O/s)
O <sub>2</sub> -concentration	21 to 100 Vol. %
Trigger sensitivity (Flow trigger)	1 to 15 L/min
Inspiratory termination criterion	5 to 75 % PIF (peak inspiratory flow)
PC-APRV (optional)	<p>Inspiratory time <math>T_{high}</math> 0.2 to 22.0 s</p> <p>Expiratory time <math>T_{low}</math> 0.1 to 22.0 s</p> <p>Inspiratory pressure <math>P_{high}</math> 1 to 95 mbar (or hPa or cmH<sub>2</sub>O)</p> <p>Expiratory pressure <math>P_{low}</math> 0 to 50 mbar (or hPa or cmH<sub>2</sub>O)</p>
O <sub>2</sub> -therapy	Constant flow Flow (BTPS) 2 to 100 L/min in increments of 1 L/min O <sub>2</sub> concentration FiO <sub>2</sub> 21 to 100 Vol% in increments of 1 Vol%
Automatic tube compensation ATC	<p>Tube type: Endotracheal tube ET or tracheostomy tube Trach.</p> <p>Inner diameter of the tube 3.5 to 12.0 mm in increments of 0.5 mm</p>

## Technical Data

### Displayed measured values

Airway pressure measurements	Max. airway pressure, plateau pressure, mean airway pressure, PEEP 0 to 99 mbar (or hPa or cmH <sub>2</sub> O)
Minute volume (MV)	Total MV, spontaneous MV 0 to 99 L/min, BTPS
Tidal volume	Inspiratory VT, expiratory VTE, VT <sub>spont</sub> 0 to 3999 mL, BTPS
Tidal volume per kg of body weight (VT / IBW)	0 to 99.9 mL/kg
Total respiratory rate	Total and spontaneous respiratory rate, 0 to 150/min
Inspiratory O <sub>2</sub> -concentration	21 to 100 % Vol.
End-tidal CO <sub>2</sub> concentration EtCO <sub>2</sub>	0 to 100 mmHg (or 0 to 13.2 Vol% or 0 to 13.3 kPa)
Breathing gas temperature	18 to 48 °C (64.4 to 118.4 °F)
Curve displays	Paw(t), Flow(t), Tidal volume (t), CO <sub>2</sub> (t)
Ventilation ratio (I:E)	1:150 to 150:1
Compliance C	0.5 to 200 mL/mbar (or mL/hPa or mL/cmH <sub>2</sub> O)
Resistance R	3 to 300 mbar/L/s (or hPa/L/s or cmH <sub>2</sub> O/L/s)
Leakage minute volume MVleak	0 to 100 %
Rapid shallow breathing RSB	0 to 9999 (1/min/L)
Special Manoeuvres	<ul style="list-style-type: none"> <li>- Intrinsic PEEP PEEPi 0 to 100 mbar (or hPa or cmH<sub>2</sub>O)</li> <li>- Exp. Hold</li> </ul>
Loops (MonitoringPlus)	<ul style="list-style-type: none"> <li>- Pressure / Volume</li> <li>- Volume / Flow</li> <li>- Flow / Pressure</li> <li>- Volume / CO<sub>2</sub></li> <li>- Ptrach – Volume</li> <li>- Flow – Ptrach</li> </ul>

### Alarms

Airway pressures	high / low
Expiratory minute volume	high / low
Tidal volume	high / low
Apnoea-alarm time	15 to 60 sec
Spontaneous breathing frequency	high
Inspiratory O <sub>2</sub> -concentration	high / low
Inspiratory breathing gas temperature	high
Inspiratory breathing gas temperature	high
EtCO <sub>2</sub>	high / low

### Performance data

Maximum (continuous) inspiratory flow	250 L/min
Valve response time T0...90	≤ 5 ms
Control principle	time-cycled, volume-controlled, pressure limited
Safety valve opening pressure	120 mbar (or hPa or cmH <sub>2</sub> O)
Emergency valve	automatically enables spontaneous breathing with filtered ambient air if air and O <sub>2</sub> supply should fail.
Automatic gas switch-over function if O <sub>2</sub> supply fails	
Output for pneumatic medication nebuliser	synchronised with inspiration
Leak compensation	synchronised patient-ventilator synchrony adjusts the flow trigger and the inspiratory termination criteria for leaks. <ul style="list-style-type: none"> <li>- tube application: up to 10 L/min</li> <li>- NIV VC-modes: up to 25 L/min</li> <li>- NIV PC-modes: unlimited</li> </ul>

## Technical Data

### Operating data

Mains power connection	100 V to 240 V, 50/60 Hz
Current consumption	max. 1.3 A at 240 V, max. 3.4 A at 100 V
Battery	internal typically 45 min (optional extension up to 5 h)
Turbine exchange interval	8 years, with no limit in operating hours during this interval
Digital machine outputs	
Digital output and input via an RS 232 C interface	
Dräger MEDIBUS and MEDIBUS.X	

### Gas supply

Air	Turbine technology (with a manufacturer guarantee of 8 years for the turbine <sup>4</sup> )
O <sub>2</sub> gas supply	3 bar (43.5 psi) – 10 % up to 6 bar (87 psi)

### Dimensions and weights

Dimensions (W x H x D)	Basic device: 460 x 383 x 364 ±2 mm (18,11 x 15,08 x 14,33 ±0,08 in) Device with Savina 300 trolley: 577 x 1295 x 677 ±5 mm (22,72 x 50,98 x 26,65 ±0,20 in) Device with Savina 300 compact trolley: 577 x 1295 x 677 ±5 mm (22,72 x 50,98 x 26,65 ±0,20 in)
Weight (basic device)	approx. 26 kg (57.3 lbs) without trolley
Diagonal screen size	12" TFT colour touch screen

<sup>1</sup> BIPAP – Trademark used under licence

<sup>2</sup> BTPS – Body Temperature Pressure Saturated. Measured values relating to the conditions of the patient lung (98.6 °F), steam-saturated gas, ambient pressure.

<sup>3</sup> 1 mbar = 100 Pa

<sup>4</sup> Limited Manufacturer Guarantee subject to conditions specified in the Instructions for Use. Applies only to devices purchased after 1/1/2015.

Some functionalities are available as an option.

## Notes

## Notes

Not all products, features, or services are for sale in all countries.  
Mentioned Trademarks are only registered in certain countries and not necessarily in the country in which this material is released. Go to [www.draeger.com/trademarks](http://www.draeger.com/trademarks) to find the current status.

### **CORPORATE HEADQUARTERS**

Drägerwerk AG & Co. KGaA  
Moislinger Allee 53–55  
23558 Lübeck, Germany  
[www.draeger.com](http://www.draeger.com)

### **Manufacturer:**

Drägerwerk AG & Co. KGaA  
Moislinger Allee 53-55  
23542 Lübeck, Germany

### **REGION EUROPE**

Drägerwerk AG & Co. KGaA  
Moislinger Allee 53–55  
23558 Lübeck, Germany  
Tel +49 451 882 0  
Fax +49 451 882 2080  
[info@draeger.com](mailto:info@draeger.com)

### **REGION MIDDLE EAST, AFRICA**

Drägerwerk AG & Co. KGaA  
Branch Office  
P.O. Box 505108  
Dubai, United Arab Emirates  
Tel +971 4 4294 600  
Fax +971 4 4294 699  
[contactuae@draeger.com](mailto:contactuae@draeger.com)

### **REGION ASIA PACIFIC**

Draeger Singapore Pte. Ltd.  
61 Science Park Road  
The Galen #04-01  
Singapore 117525  
Tel: +65 6872 9288  
Fax: +65 6259 0398  
[asia.pacific@draeger.com](mailto:asia.pacific@draeger.com)

### **REGION CENTRAL AND SOUTH AMERICA**

Draeger Panamá Comercial  
S. de R.L.  
59 East Street, Nuevo Paitilla,  
House 31, San Francisco Town  
Panama City, Panama  
Tel +507 377-9100 / Fax -9130  
[servicioalcliente@draeger.com](mailto:servicioalcliente@draeger.com)

Locate your Regional Sales  
Representative at:  
[www.draeger.com/contact](http://www.draeger.com/contact)

