

The TransWarmer® Mattress Consumables and Accessories

Critical warmth for the tiniest patient – Only nature builds a better nest.



Benefits

The TransWarmer Mattress

The TransWarmer Mattress is absolutely essential when it comes to protecting newborns from dangerous cold stress. In emergency situations, infants are quite vulnerable and at risk of hypothermia. To keep the cold and stress at bay, the TransWarmer Mattress provides up to two hours of critical warmth.

Easy-to-use and reliable

The TransWarmer Mattress is both flexible and easy to use: just firmly press the built-in trigger to activate. Its patented WarmGel® takes a mere 90 seconds to reach a temperature of 38°C (100.4 °F). WarmGel is a safe and proven latent heat store and source of steady warmth. It needs no external power supply, ensuring independence and flexibility no matter what the situation.

Comfortable

In addition to its special thermal properties, WarmGel is soft and pliable, creating a perfect "nest" for the newborn. The mattress also absorbs vibrations, thus lowering the risk of cerebral haemorrhage. This in turn allows caregivers to fully focus on the needs of the patient while meeting the challenges of transport.

Safe

Patient safety during such times is always paramount. The gel is non-toxic and does not irritate the skin or eyes, even in direct contact. Because the TransWarmer is designed for single use, it minimizes the risk of cross-contamination. It is also radio - lucent and MRI-compatible.

Warning: do not use in combination with warming therapy devices. Risk of hyperthermia!

Technical Data

Size	25 x 41 cm / 10" x 16"
Content	Sodium acetate, water, and thickening agent (all contents are non-toxic)
Operating temperature	38°C at 22°C ambient temperature
Reaches operating temperature	within 90 seconds at ambient temperatures between 18°C and 28°C
Use in transport unit, delivery room, nursery, NICU, OR and ER. (Do not use in combination with warming devices.)	

Ordering Information

TransWarmer® Mattress (6 single-use mattresses per box)	MP01414
---	---------

Notes

CORPORATE HEADQUARTERS

Drägerwerk AG & Co. KGaA
Moislinger Allee 53-55
23558 Lübeck, Germany
www.draeger.com

Manufacturer:

Dräger Medical GmbH
Moislinger Allee 53-55
23558 Lübeck, Germany

As of August 2015

Dräger Medical GmbH changes
to Drägerwerk AG & Co. KGaA

Locate your Regional
Sales Representative at:
www.draeger.com/contact



REGION EUROPE CENTRAL AND EUROPE NORTH

Dräger Medical GmbH
Moislinger Allee 53-55
23558 Lübeck, Germany
Tel +49 451 882 0
Fax +49 451 882 2080
info@draeger.com

REGION EUROPE SOUTH

Dräger Médical S.A.S.
Parc de Haute Technologie
d'Antony 2
25, rue Georges Besse
92182 Antony Cedex, France
Tel +33 1 46 11 56 00
Fax +33 1 40 96 97 20
d1mfr-contact@draeger.com

REGION MIDDLE EAST, AFRICA

Dräger Medical GmbH
Branch Office
P.O. Box 505108
Dubai, United Arab Emirates
Tel +971 4 4294 600
Fax +971 4 4294 699
contactuae@draeger.com

REGION CENTRAL AND SOUTH AMERICA

Dräger Panama Comercial
S. de R.L.
Complejo Business Park,
V tower, 10th floor
Panama City
Tel +507 377 9100
Fax +507 377 9130
contactcsa@draeger.com

REGION ASIA / PACIFIC

Draeger Medical
South East Asia Pte Ltd.
25 International Business Park
#04-27/29 German Centre
Singapore 609916, Singapore
Tel +65 6572 4388
Fax +65 6572 4399
asia.pacific@draeger.com

REGION NORTH AMERICA

Draeger Medical, Inc.
3135 Quarry Road
Telford, PA 18969-1042, USA
Tel +1 215 721 5400
Toll-free+1 800 437 2437
Fax +1 215 723 5935
info.usa@draeger.com

The quality management system at Dräger Medical GmbH is certified according to ISO 13485, ISO 9001 and Annex II.3 of Directive 93/42/EEC (Medical devices).