

Dräger X-plore® 8000 Powered Air-Purifying Respirator

Before use



D-36282-2021

1. Battery



D-1811-2014

To check the charging status of the battery, remove it from the device by pressing down the blue locking button.



D-1812-2014

Press the blue button on the battery to show the charging status. If required, charge the battery before use.



D-1813-2014

Insert the battery (blue locking button pointing upwards, contacts pointing to the device) into the battery compartment and press until it audibly clicks into place.

2. Filter



D-1815-2014

Remove the splash guard from the unit by pressing the blue filter locking button.



D-1816-2014 / D-1817-2014

Particle filter: Place the filter into the splash guard lid. Insert both at an angle into the unit. Fold it towards the device until the lid audibly clicks into place.



D-1818-2014

Gas / combination filter I: Insert the filter (with the colour marking pointing downward towards the device) at an angle into the unit.



D-1819-2014

Gas / combination filter II: Slip the splash guard lid over the filter. When correctly positioned, the lid snaps into place with an audible click.

3. Carrying system



D-1821-2014

Take the harness and button the comfort pad onto the strap if needed.



D-1822-2014

Position the connecting plate with the top edge at an angle onto the unit.



D-1823-2014

Push in the connecting plate until it audibly clicks into place on the device.



D-1824-2014

If necessary, a belt extension can be easily attached to your belt to make it longer.

4. Breathing hose



D-1825-2014

Connect the hose to the device by slightly turning the hose connector until it audibly clicks into place. Connect the hose to the face or head piece.

5. Wearing the device



D-1834-2014

Put on the carrying system and close the buckle. Tighten the straps.

6. Switching on the device



D-1835-2014

Press the green button on the control panel. The device performs a self-test. The indicator lights light up green.



D-1837-2014

The unit sets the lowest possible volume flow. Adjustment (+/-) is possible.



D-1838-2014 / D-1843-2014

Put on your head piece - see separate instruction poster.